# MEDISANA®

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GB **TENS Pain Therapy TDP**

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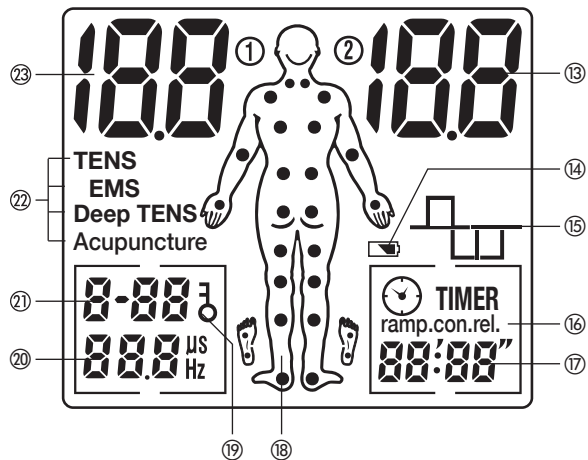
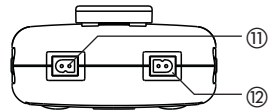
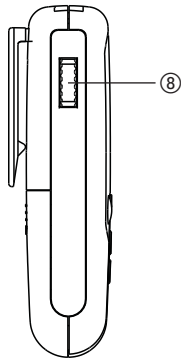
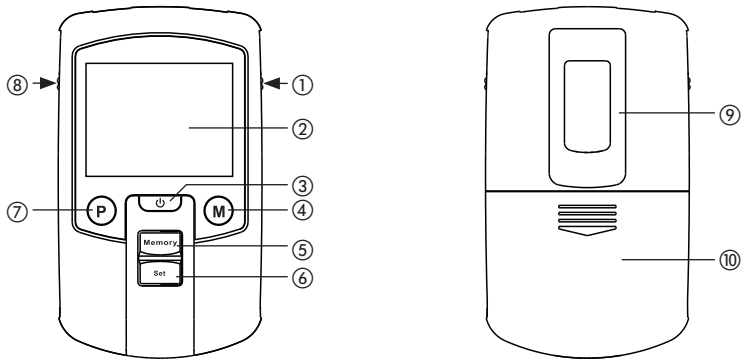
Please fold out this page and leave it folded out for quick reference.

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**GB Buttons**

- ① Current intensity control channel 2
- ② Display panel
- ③ On/off/stop button
- ④ Button M (mode)
- ⑤ Memory button
- ⑥ Set button
- ⑦ Button P (programme)
- ⑧ Current intensity control channel 1
- ⑨ Belt holder
- ⑩ Battery case
- ⑪ Channel 1 connection
- ⑫ Channel 2 connection

**Display**

- ⑬ Current intensity channel 2
- ⑭ Battery low
- ⑮ Pulse form
- ⑯ EMS ramp, continous, drop time
- ⑰ Therapy time
- ⑱ Recommended treatment points
- ⑲ Device locked
- ⑳ Frequency / Pulse width
- ㉑ Program number
- ㉒ Therapy mode
- ㉓ Current intensity channel 1

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## IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device (especially the safety instructions) and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

### Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual completely. Non-observance of these instructions can result in serious injury or damage to the device.

#### **WARNING**

These warning notes must be observed to prevent any injury to the user.

#### **ATTENTION**

These notes must be observed to prevent any damage to the device.

#### **NOTICE**

These notes give you useful additional information on the installation or operation.

Certified in accordance with Medical Product Law

Device classification: type BF

LOT number

Manufacturer

Manufacturer date

### Proper use

The Medisana TENS-/EMS device is intended for the following application fields:

- Symptomatic relief of severe chronic pain, post-traumatic pain or acute post-operative pain
- Increase of circulation in the treatment area
- Release of muscle cramps
- Post-operative stimulation of muscles to avoid thrombosis
- Prevention or inhibition of muscle loss
- Rebuilding of muscles
- Maintenance or improvement of mobility

### Contraindications

- Do not use the device if you have a pacemaker or other implants or prosthetic materials in your body.
  - If irregularities of the heart or heart diseases are known (e.g. arrhythmia or myocardial defects), you should not use the device.
  - You should consult your family doctor before using the device if you suffer from any of the following conditions or diseases:
    - Diabetes
    - Epilepsy
    - Severe circulatory disorders of the lower extremities
    - Hip or pelvis fracture
    - Cancer
  - Pregnant persons should take the necessary precautionary measures and consider their individual strength. If necessary, consult your doctor.
  - Do not treat any parts of the body that have swellings, burns, inflammation, rashes (e.g. phlebitis, thrombophlebitis, varicose veins), wounds or sensitive areas.
  - This device should not be used for symptomatic relief from local pain unless the treatment is justified or pain syndrome was diagnosed.
  - Electrodes may not be applied such that the current could run through the caroticus region (front of the throat), transcerebral (through the head) or trans-thoracic (through the chest).
- Only use the device as described in this instruction manual (also see “Application fields” on page 7). Any other use will be deemed as not in accordance with the instructions and may lead to damage to property or even injury to persons. MEDISANA AG assumes no liability for damages incurred as the result of improper use. The device is designed for private use.
  - This device is not designed to be used by persons (including children) with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety and unless they have been instructed in the use of the device by this person.
  - Children must be supervised to ensure that they do not play with the device.
  - TENS treats symptoms and suppresses the sensation of pain, which otherwise serves as a protective mechanism.
  - Electronic monitoring devices (such as ECG monitors and ECG alarms) may not function properly when electric stimulation is in use.
  - Do not use the device in the vicinity of short or microwave therapeutic devices as they may affect the output power of the device.

### Danger of injury/damage!

- Therapeutic sessions with the MEDISANA TDP do not replace medical diagnosis or treatment. Ask your doctor about each kind of pain or disease, before using the device.
- A TENS therapy should only be done according to instructions of your doctor!
- The therapy should be pleasant. If you feel pain or feel the application is unpleasant, stop it and consult your doctor.
- Do not apply the MEDISANA TDP when operating a machine or while driving a car.
- Never use the device while taking a bath or shower. Should liquids penetrate the device, immediately remove the batteries and avoid further application. Contact your retailer or your service centre.
- Do not use accessories that are not recommended by the manufacturer.
- Do not put any foreign objects into one of the outlets. Avoid contact with pointed or sharp objects. If you use the device for other purposes, the warranty expires.
- In case of a malfunction, do not try to repair the device yourself. Not only does the warranty expire, but serious danger can result (fire, electric shock, injury). Only authorised service offices should carry out a repair.
- If the device shows obvious damages on the housing or the LCD display, send it to your specialist shop or directly to the MEDISANA service centre for repair.
- Do not use damaged or defect lines to the electrodes; there is danger of electric shock.
- Leaking battery acid may lead to chemical burns. Avoid contact between battery acid with skin, eyes and mucous membranes. If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of plain water immediately and seek medical attention if needed.

- If the MEDISANA TDP device is used for commercial purposes a safety control has to be carried out at least once every two years at an authorized service centre (MPBetreibV [Ordinance on Operators of Medical Devices]).

### Notes on the electrodes

- Do not place the electrodes on areas with open skin.
- Use separate electrodes for each user.
- In case of skin irritation, stop the treatment and consult a physician.
- Take off the electrodes before taking a bath or swimming.
- After application, stick the electrodes back on the carrier paper. This way, you prolong the electrodes life.

**Thank you** for your confidence in us! The TDP pain therapy unit that you have purchased is a top quality product from MEDISANA. In order to achieve the desired effect with your MEDISANA TENS TDP pain therapy unit in the long term, we recommend that you read the following information on its use and maintenance carefully.

**Interesting information on stimulation current** Stimulation current can be used as follows:

1. to fight pain (TENS = Transcutaneous Electrical Nerve Stimulation)
2. to build up the muscles (EMS = Electrical Muscle Stimulation)
3. as electro-acupuncture.

**Pain relief (TENS)** TENS is understood as electrical nerve stimulation through the skin.

Pain is a natural and important warning signal of our body telling us that “something is wrong”. If the reason for pain is diagnosed, but cannot or can only partially be removed, a long-lasting, persistent pain may evolve (= chronic pain). This pain is therefore useless in its original form as a warning signal and can largely affect the life of the person suffering and possibly cause consequential effects such as tension, depression or reduced quality of life.

Chronic pain can be treated with different methods: e.g. with medicaments or operations, but also with alternative methods like acupuncture or TENS that are free of side-effects.

On the one hand, the TENS treatment can block pain signals by interrupting the transmission of pulses through the skin to the painful parts by means of electric impulses. The signal “pain” is not passed on to the brain and thus not felt as pain.

On the other hand, special TENS applications activate the body’s control mechanisms. Gentle electric currents make the body release so-called “endorphins” that serve to reduce pain. The pain is thus stopped, eased or removed by the body itself. This electrical stimulation cannot remove the causes or cure the disease, but change the painful feeling. Painless conditions or pain relief can help the chronically suffering person have new, positive experiences, from which he or she can gain power and energy.

**Burst TENS** Some people experience severe discomfort when exposed to frequencies under 10 Hz (that is, 10 pulses per second). The burst program generates pulse packets (“burst”) with frequencies of 80 to 100 Hz instead of a single pulse.

**Muscle building (EMS)** With different EMS applications, muscles in the body can be stimulated, which are e.g. slackened after an operation, not trained comprehensively or insufficiently or which are tensioned. In the process, electric pulses cause slight muscle contractions that can stimulate the muscular system “passively” (= without the muscles itself acting).

**Electro-acupuncture**

Acupuncture is a traditional Chinese healing method basing on the fact that there are points in the body (= acupuncture points), which, when stimulated directly, affect (e.g. animation, pain relief, increase of the energy flow etc.) certain body zones or body organs. This stimulation can be achieved with needles, massage, pressure (= acupressure) or with electric pulses, e.g. TENS. Detailed information can be found in the relevant specialist literature.

**Frequent questions and their answers****Can something happen to me, due to the current?**

No, because the current strength of the electric pulses is so low that it remains completely safe even with longer applications. However, the intensity should be adjusted to the individual sensitivity.

Therefore, start each new application with the lowest intensity level and increase it slowly according to the individual feeling. The application should never be unpleasant.

**I have no experience with stimulation current therapy yet. How can I use the device without problems?**

The MEDISANA TDP has been particularly developed to be easily used by everyone. Even without special knowledge of the subject "stimulation current therapy" you can treat yourself safely and easily with the preset programs.

**How often can I use the electrodes?**

According to the condition of your skin and if observing the application instructions you can use the electrodes up to 30 times.

**Can I use other electrodes as well?**

Yes, but you should only use original MEDISANA electrodes. You can also use duration electrodes with gel.

## Product contents

First of all, please check that the product contents are complete and that the product is not damaged in any way. In case of doubt, do not use it and contact your dealer or your service centre.

The following parts are included:

- 1 TENS unit with belt clip
- 4 electrodes
- 2 connecting cables
- 4 batteries
- 1 storage bag
- 1 instruction manual

## Packaging

The packaging can be reused or recycled. Please properly dispose of any packaging material no longer required.

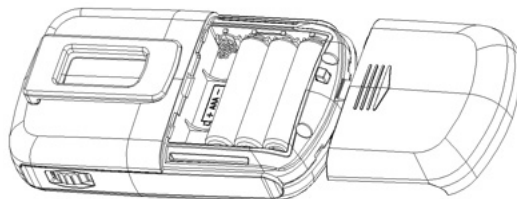
### Danger for children

- Please ensure that the packaging wrapper is kept out of the reach of children! Risk of suffocation!

## Inserting batteries

Slide the cover of the battery case on the back side of the device in the arrow direction. Insert the enclosed batteries according to the symbols (+/-) into the base of the battery case. Close the battery case again and push the cover towards the device until it snaps into position.

When changing the batteries later on, only use 1.5 V micro-type batteries (AAA/LR6) or equivalent rechargeable batteries.

### Danger of chemical burns!

#### Leaking battery acid may lead to chemical burns.

- Avoid contact between battery acid and skin, eyes and mucous membranes. If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of plain water immediately and seek medical attention if necessary. Keep batteries out of children's reach! If a battery has been swallowed seek medical attention immediately!
- Do not disassemble, recharge, short circuit batteries or dispose of them in fire! There is a risk of explosion!
- Remove batteries from the device if you do not use it for a prolonged period!
- Do not dispose of used batteries and rechargeable batteries in the household garbage, instead dispose of them as special waste or at a battery collection point in a specialist outlet!

## Application fields

The MEDISANA TDP can be used for all kinds of pain or for building up muscles, for long-term therapy as well as for acute treatment.

### PLEASE NOTE!

- Observe your doctor's precise instructions and let him show you where to apply the electrodes. For a successful therapy, the correct position of the electrodes is an important factor. Carefully write down the settings of frequency, pulse width and current selection your doctor recommended. For this, you should take your MEDISANA TDP to the doctor so that he can show you the settings.

Special textbooks provide you with detailed information concerning the subject "pain, transcutaneous electrical nerve stimulation (TENS)", concerning indications and the exact electrode application for an effective success of treatment as well as recommendations for the programs to use and notes on the settings of frequency, pulse width, time of therapy and current selection.

## Connecting the cables

At the upper side of the device there are two sockets for the connection of electrode lines. The left socket ① is for channel 1, the right socket ② is for channel 2. Connect at least one of the lines by putting the cable tightly into the socket. At the other end of the cable, you fasten one electrode each.

## Applying the electrodes

Place the electrode(s) according to your area of pain (observe your doctor's instructions).

The device must be **deactivated** before applying the electrodes!

- Before applying the electrodes, remove residual skin cream or ointment. The skin has to be dry and free of fat.
- Do not place the electrodes on injured or inflamed skin areas (wounds, pimples, skin rashes, reddening etc.).
- Always place the electrodes according to the doctor's instructions before you switch on the device.

In most cases, the electrodes are placed near the area of pain or directly on the aching spot.

- For the muscle stimulating program, observe your doctor's instructions concerning the polarity.
- If you are not sure how to place the electrode's polarity, exchange the red (positive pole, anode) and white (negative pole, cathode) connectors at the electrodes after half of the therapy time.

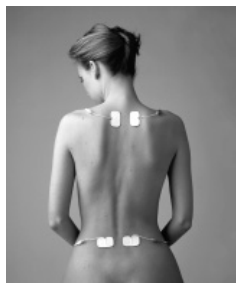
Feel for the area of pain, place one electrode directly on it and the second one to the periphery of the area of pain or, on arms and legs directly opposite.

#### Risk of electrical shock!

- Make sure that the electrodes cannot fall off during therapy. This can cause an unintentional electric shock!
- Switch off the device immediately, if an electrode comes off the skin during treatment.

#### Therapy examples:

Pain in the shoulder and pelvis area

Tension in the neck and shoulder area

For more figures on positioning the electrodes, see page 38.

Every patient reacts differently to stimulation current therapies, for this reason, positioning the electrodes and program selection may differ depending on your needs from the therapies presented here. You should take note of the parameters, which are ideal for you (electrode position, program and current intensity).

## Activation

Before you turn the device on, you have to read the safety notes (page 26) and description of the programs (starting on page 39).

To turn on the device briefly push the button ③ “⏻”. The display panel ② will light up and numbers and symbols will appear. After approx. ten seconds of key inactivity, the display light will turn off to conserve battery power. However, the symbols will remain visible. The device will automatically deactivate after approximately two minutes of key inactivity.

Push the button ③ “⏻” again to turn off the device.

### Select the therapy mode and program

MEDISANA TDP offers you four therapy modes and 100 preset programs, whose parameters are provided in the tables beginning on page 15. In addition, you can define yourself two programs in TENS mode and in one program in EMS mode (see page 35).

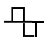
Mode	Therapy	Programs
TENS	Pain therapy	P-01 – P-10, U-01 – U-02
EMS	Muscle building	P-01 – P-77, U-01
Deep TENS	Deep pain treatment	P-01 – P-05
Acupuncture	Pain treatment, mobilisation	P-01 – P-08

- Briefly press the button ④ “M” to switch to the next therapy mode. The selected mode ⑳ blinks on the left edge of the display panel.
- Select the program ㉑ ㉒> by briefly pressing the button ⑦ “P”. Hold the button down to increase the program number ㉑ faster.

The recommended treatment points are shown on the figure ㉔ for each program. You can find examples of electrode positions on page 38.

### Select the pulse form

The device can generate different current forms for the corresponding therapies. They are classified into biphasic and mono-phasic pulses. With biphasic pulses, the current circulates between the electrodes. With mono-phasic pulses, the current only flows in one direction.

	Mono-phasic	is only used to stimulate muscles at low frequencies (up to 40 Hz).
	Bi-phasic short	is preferably used at high frequencies to relief pain. The positive (rectangular upward curve) and negative (rectangular downward curve) part follow directly after one another without interruption.
	Bi-phasic long	is used at high frequencies (starting at 60 Hz) both for pain relief and also for muscle stimulation at low frequencies (2 to 40 Hz). The positive and negative parts of the pulse are separated from one another by span width.

- Briefly pressing the button ⑤ “Memory” switches to the next pulse form. The selected pulse form appears on the display panel as a symbol ㉕.

## Select the current intensity

With the controls ⑧ (channel 1, left connection) and ① (channel 2, right connection) you set the current intensity from 1 to 60 mA in increments of 1 mA.

- Increase the current intensity: Turn the control up.
- Reduce the current intensity: Turn the control down.

The current strength appears on the display panel with the large numbers ⑫ (channel 1) and ⑬ (channel 2). When the current is activated, the therapy time ⑰ starts counting down.

Carefully increase the current intensity until you notice a tingling sensation from the applied current. Observe the instructions of your doctor or therapist in the process.

- For pain relief, only increase the intensity so that it is still pleasant. The current should not in any case be painful.
- To relax your muscles, increase the intensity until slight twitching under or around the electrodes results. In no case, should the tensing of the muscles be painful.
- Stop using the device if it is no longer pleasant or if you experience pain.

### Automatic deactivation

The device will deactivate the current from a current intensity of 10 mA if the electrodes are not properly applied to the skin or are not connected at all.

### Automatic locking

If you do not operate the unit for 10 seconds after beginning the therapy, the display panel light will deactivate and the control elements will be locked to prevent accidental changes to the settings. The lock is indicated by the key symbol ⑨ "3" next to the program number.

- To unlock the device, push the button ⑦ "P" for approximately three seconds.

## End therapy

You can interrupt therapy in progress at any time (also when the device is locked) by pushing the button ③ "⏻".

The program will stop automatically once the therapy time has elapsed.

- Turn off the device by briefly pressing the button ③ "⏻".
- Remove the electrodes, if applicable, clean them (see page 47) and stick them back on the carrier paper.

## Define TENS program

You can define a normal program (U-01) and a burst program (U-02) yourself in TENS mode for individual therapy. The following parameters are saved in the process:

- Duration of the therapy
- Frequency
- Pulse width

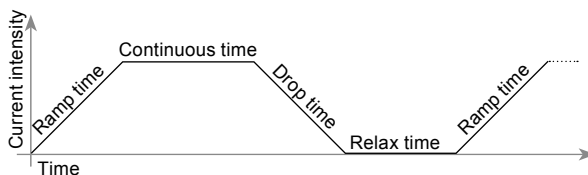
1. Select TENS therapy mode (see page 33).
2. Select one of the user programs U-01 or U-02 (see page 33).
3. Briefly press the button ④ "Set". The display for the therapy duration ⑦ will blink.
4. With one of the controls ① or ⑧ set the desired therapy duration (10 to 90 minutes).
5. Briefly press the button ④ "Set". The display for the frequency ⑩ will blink.
6. Set the desired frequency with one of the controls ① or ⑧:
  - Normal program (U-01): 2 to 120 Hz
  - Burst program (U-02): 0.2 to 5.0 Hz
7. Briefly press the button ④ "Set". The display ⑩ will now display the pulse width.
8. With one of the controls ① or ⑧, set the desired pulse width (50 to 300  $\mu$ S).
9. Briefly press the button ④ "Set".

This program has now been saved and can be used like a preset program (see page 33 and 34).

## Define the EMS program

In EMS therapy mode, you can define a program yourself and save the following parameters:

- Therapy duration
- Frequency
- Pulse width
- Increase/decrease time – Current intensity increases from zero to the defined value or drops back to zero during this time.
- Continuous time – Current intensity is held.
- Pause time – pause for recovery.

1. Select EMS therapy mode (see page 33).
2. Select the user program U-01 (see page 33).
3. Briefly press the button ④ "Set". The display of the therapy duration ⑦ will blink.
4. With one of the controls ① or ⑧, set the desired therapy duration (10 to 90 minutes).
5. Briefly press the button ④ "Set". The display for the frequency ⑩ will blink.
6. With one of the controls ① or ⑧ set the desired frequency (2 to 120 Hz).
7. Briefly press the button ④ "Set". The display ⑩ will now show the pulse width.

8. With one of the controls ① or ⑧, set the desired pulse width (50 to 300  $\mu$ S).
9. Briefly press the button ④ "Set". The display ⑯ "ramp." (ramp time) will illuminate and the ramp time will blink on the display ⑰.
10. Set the desired ramp time (1 to 10 sec.) with one of the controls ① or ⑧.
11. Briefly press the button ④ "Set". The display ⑯ "con." (continuous time) will illuminate and the continuous time will blink on the display ⑰.
12. Set the desired continuous time (1 to 20 sec.) with one of the controls ① or ⑧.
13. Briefly press the button ④ "Set". The display ⑯ "rel." (relax time) will illuminate and the relax time will blink on the display ⑰.
14. With one of the controls ① or ⑧, set the desired relax time (1 to 20 sec.).
15. Briefly press the button ④ "Set".

This program has now been saved and can be used like a preset program (see page 33 and 34).

## Delete defined programs

In order to reset the self-defined programs to the factory default, hold the button ③ "⏻" with the device on for approximately 5 seconds until a long signal is audible.

Factory settings for the programs:

### TENS

Program	Duration	Frequency	Pulse width
U-01	30 min.	50 Hz	200 $\mu$ s
U-02	30 min.	1.0 Hz	100 $\mu$ s

### EMS

Program	Duration	Fre- quency	Pulse width	Ramp/ drop	Continu- ous time	Relax
U-01	30 min.	80 Hz	150 $\mu$ s	1 sec.	8 sec.	5 sec.

**Data memory** For reference, the device saves the data from two therapies every day (e.g. in the morning and afternoon).

### Enter the date/time

The device counts the therapy days with a simple clock. To set this clock:

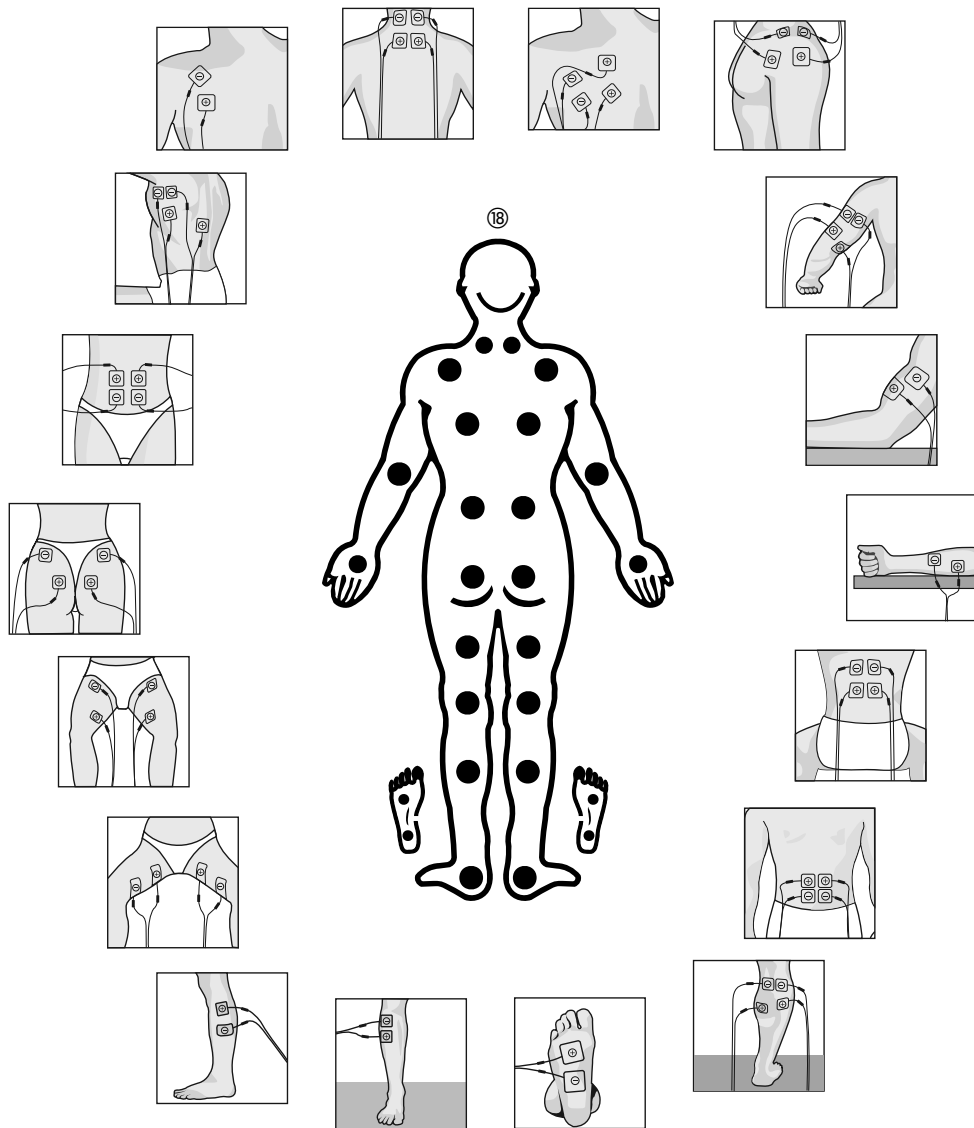
1. Activate the device with the button ③ “⏻”.
  2. Hold the button ⑥ “Set” for approximately 5 seconds until a short signal sounds. The display ⑫ will now show the therapy day, the display ⑬ will show the hour of the internal clock.
  3. With the control ⑧ set the therapy day (1 to 30).
  4. With the control ① set the hour of the day (0 to 23).
  5. Briefly press the button ③ “⏻”.
- The internal clock has been set.

### Read the memory

1. Activate the device with the button ③ “⏻”.
2. Press and hold the button ⑤ “Memory” for approximately 5 seconds until a brief signal sounds. If no data has been saved, the notice “ZERO” will briefly appear on the display ⑰ and the function is ended.
3. Otherwise the following displays will become visible:
  - ⑫ the therapy day, beginning with 1
  - ⑬ the number of the therapy session for the day (1 or 2)
  - ⑮ the pulse form
  - ⑰ the therapy duration
  - ⑱ the program number
  - ⑲ the therapy mode
4. Briefly press the button ⑦ “P” to read the current intensities. The displays ⑬ and ⑭ will now show the average current intensities for the two channels (identifiable by the channel numbers next to the numbers).
5. Briefly press the button ⑦ “P” again in order to show the therapy day again.
6. With the control ⑧, increase or decrease the therapy day.
7. With the control ①, switch between the therapy sessions 1 and 2.
8. To delete all saved therapy sessions, push and hold the button ④ “M” for approximately 5 seconds until a brief signal sounds. The status “CLR” will briefly appear on the display ⑰ and the data will be deleted.
9. To end the function, briefly press the button ③ “⏻”.

## Positioning of the electrodes

The recommended treatment points are displayed on the figure 18 for each program. The following figures provide examples of applying electrodes to these treatment points.

Program tables

TENS programs

Program	Therapy duration (minutes)	Pulse width/frequency	Current form	Neck	Shoulder	Back	Upper arm	Elbow	Arm	Waist	Stomach	Hip	Hip joint	Thigh	Knee	Calve	Feet
P-01	30	200 µs/100 Hz	Con.	•				•		•		•	•		•		
P-02	30	200 µs/35 Hz	Con.	•			•		•					•		•	
P-03	30	250 µs/2 Hz	Con.			•				•		•	•	•			
P-04	30	150 µs/100 Hz 200 µs/2 Hz	Han	•	•		•							•		•	
P-05	30	100 µs/100 Hz	Con.			•				•	•		•			•	
P-06	30	150 µs/100 Hz	Burst	•	•		•	•	•				•		•		
P-07	30	250µs/ 10 to 100 Hz/	PFM	•	•	•	•	•	•	•	•	•	•	•	•	•	•
P-08	30	50 µs/ 80 to 120 Hz	FM	•	•	•	•	•		•	•	•		•		•	•
P-09	30	150 µs/50 Hz 200 µs/2 Hz	Han	•	•		•							•		•	
P-10	30	250 µs/80 Hz	Burst	•	•		•	•	•				•		•		

- Current forms:** Con.: Continuous frequency  
 Han: Alternation between high and low frequency  
 Burst: Short pulse packets  
 PFM: Step-by-step increase/decrease of the frequency  
 FM: Stepless increase/decrease of the frequency

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**Cleaning and maintenance**

- Remove the batteries before you clean the unit.
- Never use strong cleaning agents or stiff brushes.
- Clean the unit with a soft cloth which has been lightly moistened with a mild soap sud. Never dip the unit into water for cleaning and ensure that no water gets into the unit if applicable. Only use the unit again, when it is completely dry.
- Electrodes that do not stick anymore can be cleaned under cold running water. After cleaning, leave the electrodes on the carrier paper for 24 hours to restore the adhesive strength.

**Accessories for MEDISANA TDP**

Cable set	article number 88315
4 self-adhesive multi-use electrodes, 30 x 50 mm	article number 87080
2 self-adhesive multi-use electrodes, 50 x 90 mm	article number 87100

**Technical data**

Name and model	MEDISANA TENS pain therapy unit TDP
Power supply	4 x 1.5 volt, micro-batteries (AAA, LR03) or equivalent rechargeable batteries
Channels	2
Curve form	symmetrical bi-phasic, mono-phasic
Output	constant current
Intensity	in 60 steps from 0 to 60 mA
Frequency	2 – 120 Hz, depending on the program
Pulse width	50 – 400 µs, depending on the program
Operating conditions	5 °C to +40 °C at 30 to 75 % rel. humidity
Storage conditions	-10 °C to +50 °C at 10 to 90 % rel. humidity
Dimensions (L x W x H)	approx. 115 x 67 x 35 mm (including belt clip)
Weight	approximately 99 g (without batteries)
Article no.	88310
EAN number	4015588 883101

**Electromagnetic compatibility:**

The device complies with the EN 60601-1-2 standard for electromagnetic compatibility. Visit MEDISANA's website [www.medisana.de](http://www.medisana.de) for details on this measurement data.

**Safety check (in the event of commercial use):**

The device has been calibrated by the manufacturer for a period of two years. A safety check must be conducted at least every two years if used for commercial purposes. The check is subject to charge and can be performed by a competent authority or authorised maintenance services in accordance with the "German Medical Product Operator Ordinance".

We reserve the right to make technical changes and changes to the design.

## Notes on disposal

This device may not be disposed of in the household garbage.

Every consumer is required by law to dispose of electric or electronic equipment, regardless whether or not they contain hazardous substances\*) at a collection point in his/her city or at the retail outlet so that they can be disposed of in an environmentally-friendly manner.

Remove the batteries before you dispose of the unit. Do not dispose of used batteries in the household garbage, instead dispose of them as special waste or at a battery collection point in a specialist outlet.

Contact your local administration or vendor with respect to disposal.

## Warranty and repair conditions

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for MEDISANA products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period – neither for the unit nor for replaced parts.
4. The following is excluded under the warranty:
  - All damage which has arisen due to improper treatment, e.g. nonobservance of the operating instructions.
  - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the customer service.
  - Accessories which are subject to normal wear and tear (batteries etc.).
5. Liability for direct or indirect consequential damage caused by the unit is excluded even if the damage to the unit is accepted as a warranty claim.

The service centre address is shown on the attached leaflet.

 **MEDISANA AG**  
 Jagenbergstraße 19  
 41468 Neuss  
 Germany  
 Tel.: +49 2131 3668-0  
 Fax: +49 2131 3668-5095  
 E-mail: [info@medisana.de](mailto:info@medisana.de)  
 Internet: [www.medisana.de](http://www.medisana.de)

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