# MEDISANA®

## Elektro-Akupunkturgerät ACS/ACP Electro-acupuncture device ACS/ACP

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- GB**
- 1 Acupuncture tip
  - 2 "Stimulation" display
  - 3 Stimulation button
  - 4 Intensity button (- +)
  - 5 Battery display
  - 6 On / off button
  - 7 Time display (manual, 30 sec., 60 sec.)
  - 8 Intensity display / resistance display
  - 9 Earphone connection
  - 10 Earphone (ACP only)
  - 11 Battery compartment
  - 12 Connection for external electrode
  - 13 Time selection button
  - 14 Contact electrode
  - 15 External electrode with cable (ACP only)
  - 16 Acupuncture tip for ear acupuncture

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### 1.1 Thank you very much

Thank you for your confidence in us and congratulations on your purchase! By purchasing the **ACS/ACP** Electro-acupuncture unit, you have acquired a quality product by **MEDISANA**.

This unit is intended for manual treatment by means of acupuncture.

In order to ensure that you achieve the success desired and enjoy using your **MEDISANA** Electro-Acupuncture unit **ACS/ACP** for a long time to come, we recommend that you read the following instructions carefully on use and maintenance of the device.

### 1.2 Important

**Please read the following safety information carefully before using the device and keep these instructions in a safe place for later use.**

- Only use this unit according to the intended purpose as defined in the instructions for use.
- Do not use any auxiliary parts that have not been recommended by the maker.
- Do not put any foreign objects into any of the openings.
- Avoid contact with pointed or sharp objects. The warranty is null and void if it is not used for the intended purpose.
- If the unit is to be used by or on people who are handicapped in any way or in the presence of children, this must always be done under supervision. Medical products are not designed to be used as a toy!
- Do not attempt to repair the unit yourself in the event of any defects. Not only does this render the warranty null and void but you can also run the risk of serious hazards (fire, electric shock, injury). Only have repairs carried out by authorised service points.
- Never use the device in the bathtub or shower. If liquid should ever happen to enter the device, the batteries must be removed immediately and the device should no longer be used. Contact your dealer or our service point.
- Batteries must be deposited in special collection containers or returned to a specialized dealer. Never put these in the household trash.

### 1.3 Important recommendations concerning your health

- Consult your doctor first before using this unit if you have diabetes or if you are suffering from any other illness.
- Pregnant women should note the important precautionary measures and the amount of stress they can take, consult your doctor first if necessary.
- Do not treat any parts of the body that have swellings, burns, inflammation, peeling skin, wounds or other sensitive points.
- The treatment should be pleasant. Stop at once and consult your doctor if you feel pain or if the treatment seems unpleasant.

## 1.4

### Caution! Characteristic features of the MEDISANA ACS/ACP

- Treatment with the **MEDISANA ACS/ACP** cannot replace diagnoses and treatment by a physician. Consult your doctor about any type of pain or disease before using the device.
- Do not use the device if you have a pacemaker or other implantation.
- The electro-stimulation must not be carried out while operating a machine or driving an automobile.
- If you are known to have irregularities of the heart or heart disease (e.g. arrhythmia or myocard disorders), you should not use the device.
- If electrodes are used, these must not be applied to injured areas of the skin. Remove the electrode before swimming or bathing.
- The contact electrode tip and the acupuncture tip is made out of gold plated brass. In isolated cases allergic reactions of the skin can occur.
- If the skin becomes irritated, discontinue the treatment and consult a physician.
- Do not use the device in the vicinity of the throat.
- An acupuncture treatment should be applied only on the points indicated in the accompanying book. Ask your doctor or non-medical practitioner for additional information.

## 2.1

### Scope of supply and packaging

Please check first of all that the unit is complete.

The following parts are included as standard:

- Electro-Acupuncture Unit **ACS/ACP**
- Acupuncture tip
- 2 button-type lithium batteries CR2032, already inserted
- Earphone (ACP only)
- External electrode with cable (ACP only)
- Accompanying book
- Operating instructions

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

#### WARNING

**To avoid risk of suffocation, keep packaging film away from children!**

## 2.2

### Valuable information about acupuncture

Acupuncture is an ancient Chinese art of treatment, in which needles are inserted into precisely defined areas of the body, the so-called acupuncture points. These are located along lines of the body called meridians.

By means of these acupuncture points, energetic disorders in the organism are equalized, and specific bodily organs or zones can be stimulated, thereby also stimulating their function and vitality. This stimulation can also be achieved by pressure (acupressure) or harmless electrical impulses such as with the **MEDISANA ACS/ACP** at the respective acupuncture points.

Acupuncture is applied especially in the case of pain syndromes and many types of dysfunctions.

### 2.3 Insert/replace batteries

When the battery energy begins to be expended, the red LED **5** begins to blink. In order to ensure proper functioning of the device, change the batteries. To do this, push the battery compartment lid **11** on the bottom of the device away toward the back. Remove both batteries and replace them with two new batteries of the same type (two 3V Lithium CR 2032). When inserting them, make sure that the + ends face the bottom of the device (the side with the type plate). Close the battery compartment.

#### BATTERY SAFETY INFORMATION

- Keep away from children.
- Do not short-circuit.
- Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop.
- Do not recharge.
- Do not throw into fire.

### 3.1 Areas of application

The **MEDISANA ACS/ACP** can be used in three ways:

*Search mode:* this locates the acupuncture point.

*Manual treatment:* no time limit for the treatment.

*Treatment with automatic timer:* 30 seconds or 60 seconds of treatment on the acupuncture point.

Turn on the device by pressing the button **6**. The switched-on condition is indicated by an audio signal and the illuminated LED lamp **7** above the time interval button **13**.

#### 3.1.1 Operation

Hold the device as shown in the photo.

Hold the **MEDISANA ACS/ACP** so that you are touching the contact electrode **14**. A complete circuit is necessary for operation. The contact electrode **14** forms one pole (counter electrode) and the acupuncture tip **1** the other pole (active electrode).

When treating another person it is likewise important to keep this circuit complete, i.e. by touching the person being treated with your free hand or by using the external electrode (see "External electrode").

### 3.1.2 Search mode

Apply the tip to the skin with light pressure - in the approximate area where the acupuncture point should be located (see accompanying book). When you begin searching, the audio signal will become brighter, and one or more LEDs will light up on the display the closer you come to an acupuncture point. Search for the point with the tip until the audio signal is at its brightest and the most diodes **8** are illuminated.

#### NOTE

**Sometimes acupuncture points are located very close to one another. In order to find the right point for the desired treatment, always refer to the accompanying acupuncture book or specialized literature.**

### 3.1.3 Manual treatment

Once you have found the acupuncture point, press the stimulation button **3** with your index finger and keep it depressed. The audio signal ceases and the LED **2** begins blinking; at the same time the first LED in display **8** blinks continuously. You are now treating the acupuncture point with the lowest electric current (basic setting). If you wish to increase the intensity, it is recommended to keep the tip **1** on the acupuncture point and to change the intensity setting (see also "Adjusting the intensity"). As soon as you release the button **3**, the stimulation ceases and the device returns to search mode. Therefore you yourself determine how long the electrical impulses are transmitted.

### 3.1.4 Treatment with automatic timer

By means of an internal clock the **MEDISANA ACS/ACP** can be set for 30 seconds or 60 seconds of stimulation using button **13**. The set time is retained until it is changed by pressing the button **13** or until the device is switched off. In the basic setting, as soon as you turn on the device, the manual mode is always activated first.

Before searching for the acupuncture point, set the desired time by pressing the button **13**: press once for 30 seconds, and press again for 60 seconds. The LED **7** displays the respective setting.

Search for the acupuncture point (see "Search mode") and briefly press the stimulation button **3** once. The LED **2** begins to blink. Since you do not have to keep the stimulation button **3** depressed, you can comfortably hold the device on the acupuncture point with light pressure.

After the set time has expired the device returns to the search mode. Since the time setting is retained, you can stimulate the point again for the same set time by again briefly pressing the stimulation button **3**.

### 3.2 Adjusting the intensity

After switching on the device, the **MEDISANA ACS/ACP** is always set at the lowest level of intensity (basic setting). For certain acupuncture treatments, however, higher intensities may be necessary.

You can change the effective intensity before or during stimulation. It is recommended, however, not to adjust the intensity until stimulation is applied, since different areas of the body vary in sensitivity and also because different people experience the electric impulse subjectively.

When the device is on, press the intensity button **4** on the plus sign (+) repeatedly until you have reached the desired intensity. In the display **8** you will see that the light display moves one step further each time the button is pressed.

When the last LED is illuminated, the device is set at the highest intensity. If you press the intensity button **4** on the minus sign (-), the intensity is reduced step by step down to the lowest intensity (foremost LED).

#### NOTE

**The intensity is experienced very differently at different points of the body. Before changing the acupuncture point, it is therefore recommended to always reset the intensity to the lowest level!**

### 3.3 Using the unit's accessories

With the **MEDISANA ACS/ACP** you will receive an acupuncture tip **16**, an earphone **10** (ACP only) and an external electrode with cable **15** (ACP only).

For the **MEDISANA ACS**, the **earphone** and **external electrode** (item no. 81650) are available as an accessory set.

#### Acupuncture tip

The additional acupuncture tip **16** has a finer and very narrow tapered end, enabling even more efficient stimulation of the acupuncture point. This can be especially useful in searching for, finding and stimulating points located very close together, e.g. in the ear. Simply place the tip on the device. Make sure that it is seated securely and has contact with the device.

#### Earphone (ACP only)

The earphone **10** can be connected to the **MEDISANA ACS/ACP** by means of the jack **9**. Now the acoustic signal can only be heard through the earphone.

#### NOTE

**Remember to switch off the device after completion of the treatment. If you have worked with the earphone and set it aside with the device, you can tell that the device is switched on only by means of the illuminated LED **7**.**

**The battery consumption increases if you leave the MEDISANA ACS/ACP switched on unnecessarily.**

**External electrode with cable (ACP only)**

Plug the external electrode **15** into the connection jack **12**. For self-treatment, fasten the electrode around your own wrist or arm, for treatment of another person around their wrist or arm. Now you can hold the **MEDISANA ACS/ACP** as you wish, without having to concentrate on keeping contact with the contact electrode **14** or - when treating someone else - without having to touch the person being treated more than necessary. The completed circuit is ensured by attaching the external electrode. This enables more convenient treatment.

**NOTE**

**You can also improve the contact with the external electrode by slightly moistening the electrode before applying it to the skin.**

**4.1  
Frequently asked  
questions and  
their answers****Why does my device sometimes indicate no acupuncture points?****1. For self-treatment**

Do you have contact with the contact electrode **14**? The electric circuit must always be complete. Hold the **MEDISANA ACS/ACP** in such a way that at least one finger is touching the contact electrode.

**2. For treatment of others**

Have you completed the electric circuit, i.e. contact with the person being treated? For treatment with the use of the external electrode: Is the external electrode properly applied, i.e. securely on the non-oily skin? Is the connection cable connected tightly in the jack **12**? Here one must also make sure that the electric circuit is always complete.

**3. For treatment with the acupuncture tip **16****

Does the attached tip have contact with the device? If necessary, remove it once and replace it, turning it in order to restore contact.

**How do I turn off the sound?**

Plug in the earphone. Now you can no longer hear the sound in the treatment environment, but only through the earphone. The display of the acupuncture points is still indicated additionally by means of the optical signals **8**.

**Can the electricity harm me?**

No, because the intensity of the electrical impulses is so low that it is fully harmless even when applied over a longer period of time. The intensity should, however, be adjusted depending on the sensitivity of the application point. For example, the skin in the facial area is more sensitive than on the hand. For each new stimulation of a point, you should therefore begin with the lowest level of intensity, increasing it gradually according to individual preference (see also: "Adjusting the intensity"). The treatment should never be unpleasant.

**What intensity should I select?**

Please adjust the intensity in such a way that the sensation is pleasant. Under no circumstances should the experience be painful, although the impulses should be noticeable. The intensity also depends on where you apply acupuncture.

**4.2  
Cleaning and  
maintenance**

- Remove the batteries before you clean the unit.
- Never use strong cleaning agents or stiff brushes.
- Clean the unit with a soft cloth which has been lightly moistened with a mild soap. Never dip the unit into water for cleaning and ensure that no water gets into the unit at any time. Only use the unit again, when it is fully dry.
- If there are separate heads for the unit, then clean them as necessary with a small soft brush. First allow the heads to dry off completely before you use them again on the unit.

**4.3  
Disposal**

This product must not be disposed together with the domestic waste.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

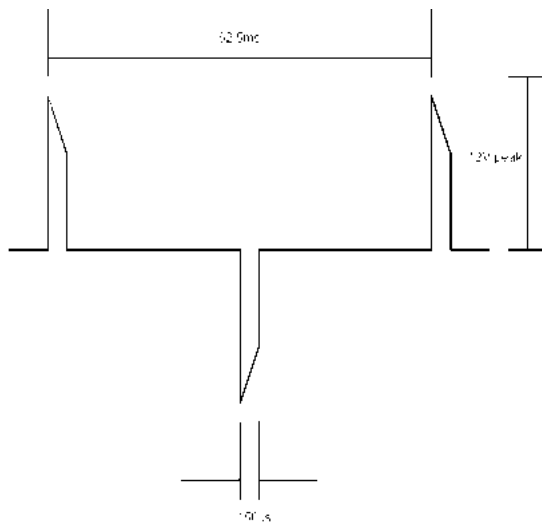
Please remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop.

Consult your municipal authority or your dealer for information about disposal.

**4.4 Specifications**

Name and model	: <b>MEDISANA</b> Electro-Acupuncture Unit <b>ACS/ACP</b>
Excited voltage	: max. 12 V at 500 ohm / max. 100 V no load
Strength of current	: max. 48 mA peak at 500 ohm
Excitation time	: manually / 30 s / 60 s
Signal duration	: 150 µs
Pulse rate	: 16 Hz
Power supply	: 6 volt= (2 X 3volt lithium battery type CR2032)
Durability of batteries	: approx. 2200 cycles of 30 s length
Classification according to MPG	: Ila
Protection class	: BF 
Dimensions	: approx. 175 x 18 x 40 mm (L x W x H)
Weight	: approx. 45 g incl. batteries
Article no. (ACS)	: 81502
EAN-Number (ACS)	: 4015588815027
Article no. (ACP)	: 81600
EAN-Number (ACP)	: 4015588816000

*Output curve shapes at 500 Ohm*

**CE 0297**

**In accordance with our policy of continual product improvement, we reserve the right to technical and design changes without notice.**

## 5.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to send in the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

### **MEDISANA AG**

Itterpark 7-9

D-40724 Hilden

Tel.: +49 (0) 2103 / 2007-60

Fax: +49 (0) 2103 / 2007-626

eMail: [info@medisana.de](mailto:info@medisana.de)

Internet: [www.medisana.de](http://www.medisana.de)

The service centre address is shown on the last page.

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