## Medinose Plus

Anti-Allergiegerät

Anti Allergy Unit

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## Thank you

Thank you for your confidence in us and congratulations on your purchase! By purchasing the **Medinose Plus** anti allergy unit, you have acquired a quality product by **MEDISANA**.

This unit is designed for phototherapy of allergic symptoms and over-sensitivity of the nasal mucosa.

To ensure that you are able to obtain the results you are looking for and can enjoy using your **MEDISANA** anti allergy unit **Medinose Plus** for a long time to come, we recommend that you read with care the following information about using and caring for the device.

## 1.1 Important

**Read the following safety information carefully before using the device and keep these instructions in a safe place for later use.**

- The appliance is licensed for home therapy and must not be used for commercial purposes.
- Therapys with the **Medinose Plus** do not replace medical diagnosis or treatment.
- Always seek the advice of a doctor in case of doubt and follow his instructions for treatment.
- Never change the quantity of medication prescribed by your doctor!
- The appliance will have limited or no effectiveness in the case of chronic inflammations of the nasal mucosa, deformations of the nasal septum or polyps.
- Applications with the **Medinose Plus** are generally perceived as pleasant. Cut short the application immediately if you experience sudden pain or the application becomes unpleasant and consult your doctor.
- Children may only use the appliance under adult supervision. Medical products are not toys!
- The unit contains small parts which could be swallowed by children. Please do not leave children unsupervised whilst using the **Medinose Plus** anti-allergy unit.
- Keep the battery out of the reach of children. The battery is not rechargeable. Do not short-circuit the battery or throw it into a fire. Hand it in to your dealer or a local collection point for recycling.
- Leaking batteries can cause chemical burns on contact with skin. If battery fluid comes into contact with skin, rinse immediately with plenty of water and seek medical advice if necessary.
- Only use the appliance for the purpose defined in the instructions for use.
- The right to claim under the warranty will expire in the event of misuse.
- Only use nasal applicators from **MEDISANA**.

## 1.2 Important considerations

- The device may not be used whilst driving a car or whilst operating and controlling machinery.
- Never use the device whilst taking a bath or shower.
- Protect the device from moisture.
- If liquid should get into the device, remove the battery immediately and discontinue use. Should this happen, contact your specialist dealer.
- Do not expose the device to direct sunlight.
- Do not use the device if it is not working correctly, if it has been dropped or has fallen into water or has been damaged.
- Do not attempt to repair the device yourself in the event of a malfunction since this will invalidate the warranty. Repairs should only be carried out by authorised service centres.
- When the device is not in use it should be stored in a clean and dry place, ideally in its storage box.
- Remove the battery and store it in a cool place if the unit will not be used for some time.
- Should you give this massage device to another person, it is vital that you also pass on these instructions for use.

## 1.3 Scope of delivery and packaging

Please check first that the unit is complete.

The following are included with the **Medinose Plus**:

- 1 **MEDISANA** anti allergy unit **Medinose Plus**
- 1 9 volt monobloc battery 6LR61
- 1 nasal applicator
- 1 storage box
- 1 pollen count calendar
- 1 instructions for use

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

### WARNING

**To avoid risk of suffocation, keep packaging film away from children.**

### **2.1 Allergies – Why are they becoming more frequent?**

Illnesses caused by allergies such as hay fever and allergic reactions to house dust and animal hair have almost doubled in the last few years. It is not yet medically clear why more and more people are suffering from allergic symptoms. Exaggerated hygiene in childhood may hamper the development of the immune system and favour allergies. It is certain, however, that along with heredity increased environmental pollution plays a significant role in the development of allergies.

### **2.2 Hay fever – A trifling complaint?**

Hay fever, otherwise known as allergic rhinitis, is not actually a fever and does not necessarily have anything to do with hay. It is not caused by a virus but by pollen. Hay fever should always be taken seriously. The allergy can spread to the lower respiratory passages, leading to chronic disorders and finally to asthma. Your doctor can provide you with a diagnosis and the correct individual advice for you.

### **2.3 Hay fever – How it arises**

It is our immune system's job to fend off and render harmless any contaminants that invade our bodies. In the case of an allergy, the immune system over-reacts to these contaminants. Even the tiniest amounts of allergens, such as flower pollen for example, are enough to induce the typical hay fever symptoms:

- Agonising desire to sneeze combined with violent sneezing attacks,
- Swollen, itchy nasal mucosa,
- Itchy, watery, swollen eyes.

The reason for this is an over-reaction of our defence mechanism. As soon as the flower pollen touches the mucous membranes of the nose and eyes, the body mobilises all its defences. Too many antibodies are created. These sit on the so-called mast cells. The mast cells in turn release Messenger substances (primarily histamine) that lead to the typical miseries of hay fever.

### **2.4 What can help to combat this over-reaction?**

Of course, the easiest way of preventing an allergy is to consistently avoid all substances that trigger allergies. However, in practice this is hardly possible as one can come across allergens not only outside but also at home (due to house dust) or in foodstuffs.

In addition to drug treatment using antihistamines and cortisone, inoculations and acupuncture, phototherapy is also used successfully. In this case red light rays of one wavelength (approx. 660 nm) are generated by special light-emitting diodes to reduce or to eliminate the allergic symptoms and the over-sensitivity.

## 2.5 How the Medinose Plus uses phototherapy to help you

Phototherapy is a new type of allergy treatment without drugs. Diodes which are capable of emitting red light with a wavelength of 660 nm have been developed for the **Medinose Plus**. Alleviation of allergic symptoms and lessening of the over-sensitivity of the nasal mucosa sets in at this wavelength.

## 2.6 The benefits of your photo- therapy with Medinose Plus:


- Phototherapy is a drug-free treatment.
- The highly effective application lasting only approx. 4 minutes can be carried out practically anywhere.
- The treatment has no effect on other organs or body parts due to its use locally.

The typical symptoms of hay fever were significantly reduced in clinical test using the **Medinose Plus**. Some symptoms were alleviated or even eliminated completely in a matter of only a few hours. In the majority of patients it was possible to identify a clear improvement in the condition within two weeks.

## 3.1 Inserting / removing batteries

To insert the 9 Volt battery included with the unit, open the battery compartment on the back of the unit. Press down on the area marked on the lid of the battery compartment and simultaneously slide it in the direction of the arrow. Insert the battery in the position shown inside the battery compartment. Replace the battery compartment lid and slide it upwards until it clicks audibly into place.

After 100 applications, the green control LED **6** will flash after pressing the ON/OFF switch **7** if the battery power is no longer sufficient for further use. It is then time to replace the spent battery.

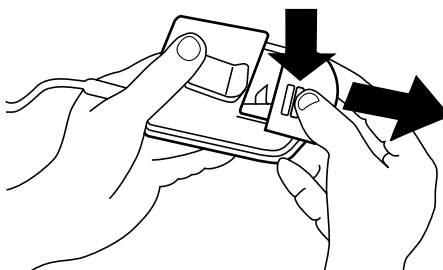
Remove the spent battery and insert a new 9 volt monobloc battery 6LR61 (in the position shown inside the battery compartment). Then replace the battery compartment lid and slide it upwards until it clicks audibly into place.

#### BATTERY SAFETY INFORMATION

- Keep away from children.
- Do not recharge.
- Do not short-circuit.
- Do not throw into fire.
- Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop.

### 4.1 Use of phototherapy using the Medinose Plus

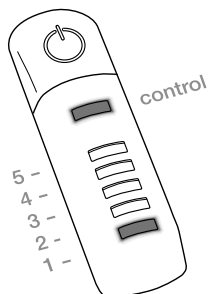- Insert the plug **9** of the nasal applicator into the socket of your **Medinose Plus** appliance **10**.
- The nasal applicator **3** must be spread slightly apart to allow the LEDs to be inserted in the nostrils. Insert the LEDs as far as possible into the nostrils. The nasal applicator should nevertheless sit comfortably.
- Press the ON/OFF button **7** to start the treatment and begin light emission. The green pilot LED **6** on the front of your **Medinose Plus** will be lit constantly during use.
- At the same time the bottom red LED assigned to the number **1** will light up. The time elapsed will be displayed by the appropriate LED in each case.
- Light emission will be ended automatically after approx. 4 1/2 minutes. The green pilot LED **6** and the LED display **5** will go out. A short acoustic signal (beep) will also sound. This indicates that the application is over.
- Now remove the nasal applicator **3** from the nostrils.

#### NOTES

- If the green control LED **6** lights up and a beep is heard at the same time, this indicates that the connector is not plugged in correctly or there is insufficient charge in the battery for an application.
- If the green pilot LED **6** neither lights up nor flashes this indicates that the battery has been incorrectly inserted or is completely discharged.

#### 4.2 Duration and Frequency of Application

- An application with the **Medinose Plus** appliance only lasts approx. 4 minutes. During this time, the LEDs radiate red light having a wavelength of approx. 660 nm and a defined intensity.
- The automatic shut-off ensures that the duration of treatment always remains the same and prevents the nasal mucosa from being overloaded.
- The **Medinose Plus** should be used three times a day at the beginning of treatment. The frequency of application may be reduced when symptoms have been alleviated.
- Treatment should be spread over the course of a day. The first application should be carried out either before or after breakfast and the last in the evening before going to bed whilst the third application can be carried out during the day simply by taking the appliance with you and using it when the opportunity arises. The practical belt clip  on the back of the unit makes it easy to wear.

#### NOTE

**The applications should be continued for at least one week and preferably two to achieve the desired treatment result.**

#### 4.3 Preventive Treatment

Your **Medinose Plus** appliance may be used not only in the acute phase of the illness but also as a preventive measure. You will most likely know the period during which your allergy starts or the type of pollen that causes it.

Begin prophylactic treatment a good week before the pollen that triggers your allergy is forecast. Even at this stage you should carry out an application two to three times a day depending on the severity of your allergy.


#### 4.4 Phototherapy – safe and effective

The red light used for phototherapy with **Medinose Plus** has no negative effects on the nasal mucosa. Even with therapy continued over a longish period, there is no fear of any structural impairment of the tissue when no more than three applications per day are administered. If your doctor suggests medical treatment due to the severity of your allergy, it may be combined with phototherapy without hesitation. There are no known interactions with anti-allergic drugs such as antihistamines or corticosteroids.

### 5.1 Cleaning and Care

- You may only clean the unit yourself.
- Clean both LEDs after each use with a damp cloth and a mild soap.
- Do not use aggressive detergents or stiff brushes under any circumstances.
- Do not immerse either the appliance or the cable with the LEDs in water. If liquid has managed to get into the appliance, remove the battery immediately and contact your specialist dealer.
- The nasal applicator may be disinfected with an alcoholic disinfectant, e.g. 70% ethanol.
- You may purchase additional nasal applicators (Article no. 45005) from your specialist dealer so that your **Medinose Plus** appliance can be used by several people under hygienic conditions.
- Remove the battery from the appliance if you do not intend to use it for a longish period. This will prevent the battery from leaking.
- No further maintenance or checking is necessary if the appliance is used in accordance with these instructions for use.

### 5.2 Notes on Disposal

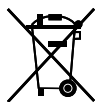Dispose of your old electrical unit in an environmentally acceptable manner when it has reached the end of its service life!

Packaging is re-usable or may be re-cycled; metal parts must be disposed of as scrap metal; plastics, electrical and electronic components must be disposed of as electro-scrap.

Please remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop.

Consult your local council or your dealer for recycling information.

### 5.3 Directives / Norms

The **Medinose Plus** appliance from **MEDISANA** carries the CE symbol (conformity symbol) "CE 0473" confirming the appliance's conformity to "Guideline 93/42/EEC of the Council dated 14 June 1993 governing medicinal products".

The EC importer as defined by EC Guideline 93/42/EEC is  
Trebs B.V., NL-6467 JE-Kerkrade

The corresponding conformity declaration is available at **MEDISANA** premises.

**Appliance classification:** Type BF 

**5.4  
Technical Data**

|                          |   |
|--------------------------|---|
| Name and model           | : <b>MEDISANA</b> anti allergy unit <b>Medinose Plus</b>          |
| Application              | : Phototherapy for allergic reactions of the nasal mucosa         |
| Voltage supply           | : 1 9 volt monobloc battery 6LR61                                 |
| Service life of battery  | : approx. 100 applications  |
| Duration of application  | : 4 1/2 minutes +/- 10%   |
| Automatic shut-off       | : after recommended duration of application                       |
| Frequency of application | : Maximum 3 applications per day                                  |
| Light emission           | : Red light of approx. 660 nm at low intensity of 6 mW per diode  |
| Storage conditions       | : -20 °C to +50 °C, maximum 85% relative humidity, non-condensing |
| Operating conditions     | : +10 °C to +40 °C, maximum 70% relative humidity, non-condensing |
| Dimensions               | : approx. 90 x 65 x 30 mm   |
| Weight                   | : approx. 120 g including battery                                 |
| Article no.              | : 45040   |
| EAN no.                  | : 4015588450402   |
| Spare applicator         | : Article no. 45005   |

**CE 0473**

**In accordance with our policy of continual product improvement, we reserve the right to technical and design changes without notice.**

### 6.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to send in the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA HEALTHCARE UK LTD.  
City Business Centre 41 St.  
Olav's Court Surrey Quays  
London SE16 2XB  
Tel.: + 44 / 207 - 237 88 99  
Fax: + 44 / 207 - 252 22 99  
eMail: info@medisana.co.uk  
Internet: www.medisana.co.uk

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D&L Marketing Oy
ALERE Electronics AB
**MEDISANA AG**