MEDISANA®

Medistim/Medistim-Duo

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Thank you very much

Thank you for your confidence in us and congratulations on your purchase! By purchasing the **Medistim** pain relief plaster, you have acquired a quality product by **MEDISANA**.

To ensure that you are able to obtain the results you are looking for and can enjoy using your **MEDISANA** pain relief plaster for a long time to come, we recommend that you read with care the following information about using and caring for the device.

1.1 Important

Read the following safety information carefully before using the device and keep these instructions in a safe place for later use.

- The device should not be used by the following patients:
 - pregnant women during the first 12 weeks of pregnancy and generally speaking not on the stomach at all during the entire pregnancy,
 - patients with pacemakers or extreme cardiac irregularities
 - patients with metal implants
 - diabetics and patients with abnormally high blood pressure
 - cancer patients, patients with severe inflammations, acute illnesses accompanied by fever, infections, inflammatory skin diseases, leprosy sufferers
 - patients under 16 years of age and not at all by children under 12.
- Children may not use the device. Medical products are not toys!
- Consult your doctor if you suffer from epilepsy or certain disabilities.
- Always seek the advice of a doctor in case of doubt and follow his instructions for treatment.
- This device may not be used to treat the following areas of the body: the carotid sinus nerve, the laryngeal and pharyngeal muscles.
- The electrodes may not be placed on the following areas of the body: on the head, the neck, in the vicinity of the heart, on the genitals, on areas of scarring if the operation was performed less than 10 months previously, on inflamed or injured areas of skin or varicose veins.
- Pain relief treatment following injury or operations may only be carried out under the supervision and instruction of a doctor or physiotherapist.
- The **Medistim**'s electric pulses may be unpleasant but should not be painful. If treatment causes pain, first reduce the intensity and if you continue to experience pain, stop the treatment and consult your doctor for advice.

1.2 Important considerations

- Take care to ensure that the two ends of the electrode pads never come into contact. This could lead to short-circuiting of the device.
- Do not switch the device on when it is not positioned on the body.
- Do not place the electrodes on the body if it is wet.
- Do not use the device whilst taking a bath or a shower.
- The electrodes should not be applied to the musculature above the stomach within 90 minutes of eating.

- The device may not be used whilst driving a car or whilst operating and controlling machinery.
- Never remove the device from the skin whilst it is still switched on.
- The device can be switched off at any time if necessary. Press – /OFF button ② until red function display ③ shines more brightly. Press the button once more and the device is switched off.
- Do not operate the device in the vicinity of high-frequency transmitters, e.g. medical microwave and short wave transmitters or radio transmitters. They may impair the device's ability to function.
- Do not expose the electrode pad to direct sunlight.
- Protect the device from moisture.
- If liquid should get into the device, remove the battery immediately and discontinue use. Should this happen, contact your specialist dealer or contact us directly.
- Never use the device in rooms where aerosols (sprays) are used or pure oxygen is being administered.
- Use the device only according to its intended purpose as specified in the instruction manual.
- The warranty will be invalidated if the device is used for purposes other than those for which it is intended.
- The device may only be used in conjunction with the accessories (electrodes) recommended by the manufacturer.
- Do not use the device if it is not working correctly, if it has been dropped or has fallen into water or has been damaged.
- Do not attempt to repair the device yourself in the event of a malfunction since this will invalidate the warranty. Repairs should only be carried out by authorised service centres.

1.3 Scope of delivery and packaging

Please check first that the unit is complete.

The following are included with the **Medistim**:

- **MEDISANA** pain relief plaster
 - 1 Electrode pad
 - 1 Coin cell battery 3 V (CR 2032)
 - 1 Storage box
 - 1 Instructions for use

The following are included with the **Medistim-Duo**

- 2 **MEDISANA** pain relief plasters
 - 4 Electrode pads
 - 2 Coin cell batteries 3 V (CR 2032)
 - 1 Storage box
 - 1 Instructions for use

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

**WARNING**

To avoid risk of suffocation, keep packaging film away from children.

2.1 Nerve and Muscle Stimulation using the Pain Relief Plaster

Your **Medisana** pain relief plaster **Medistim** is a TENS/EMS device.

TENS = **T**ranscutaneous **E**lectric **N**erve **S**timulation

EMS = **E**lectric **M**uscle **S**timulation

It provides treatment in the low frequency range using the TENS/EMS method. Using this method, weak electrical pulses are conducted through the skin using the pain relief plaster thus stimulating muscles, nerve fibres and acupuncture meridians in order to utilise the body's own pain relief mechanisms.

2.2 Possible Treatment Options

The treatment may be used:


- for local pain relief
- relief for symptomatic, chronically stubborn pain
- in assisting the treatment of post-operative and traumatic acute attacks of pain and phantom pain
- in assisting the treatment of headaches, tension in the upper back and nape of neck area, painful shoulder and arm tension
- in assisting the treatment of tennis elbow, sciatica and joint pain
- for circulation problems and menstrual pain
- in assisting the treatment of painful ligaments and muscle injuries
- for strengthening and toning the muscles
- for bruising, strains and sprains

3.1 Prior to Use

1. The round insulating film must be removed from the battery compartment before the device is used for the first time. To do this, open battery compartment ⑨ (see below, Section 3.2). Remove the battery and take the round insulating film out of the battery holder. Insert the coin cell battery carefully and push the compartment lid back in place.
2. Attach electrode pad ④ to the device by pressing contact plug ⑤ and contact jacks ⑧ into one another (see front fold-out page).
3. Do not remove the protective plastic film from the underside of electrode pad ④ until the device and the electrode are firmly attached to one another, and only remove it when the device is to be used. The protective plastic film serves as additional protection for the electrode pad during storage in the storage box.
4. The area of skin to which electrode pad ④ is to be attached for treatment should be clean and free from grease to ensure that the electrode pad retains its adhesion for a long time.
5. Remove the protective plastic film from the underside of the electrode pad so that it is ready for use. Hold the electrode firmly by its edge and remove the film by peeling it off evenly.

3.2 Inserting / removing batteries

Open the battery compartment by sliding the lid and compartment in opposite directions. Remove the spent battery and replace it by inserting a new coin cell battery (CR 2032) into the battery compartment with the + pole uppermost (see diagram). Slide the lid back onto the lower casing and close the battery compartment.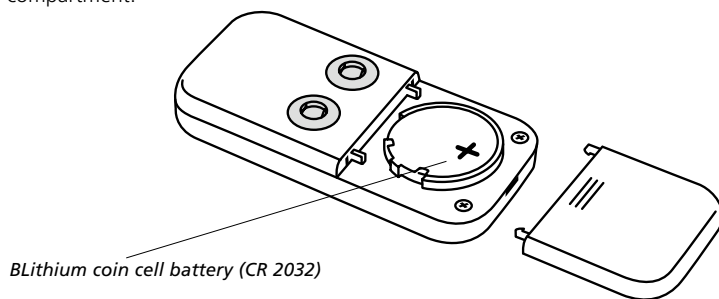

BATTERY SAFETY INFORMATION

- Keep away from children.
- Do not short-circuit.
- Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop.
- Do not recharge.
- Do not throw into fire.

4.1 Use and Function

1. Familiarise yourself with the safety notes in Sections 1.1 and 1.2 on pages 10 – 11 and then proceed as described in Section 3.1 points 1 – 5.
2. Place electrode pad **4** on the area of skin selected and smooth it down with firm pressure. Switch the device on by pressing ON/ + button **1**. Function display **3** lights up brightly. By pressing ON/ + button **1** again, the device starts to work at the lowest level of intensity. The device has 10 levels of intensity for strength of treatment ranging from 0 - 60 mA. Select these intensity levels by simply pressing ON/ + button **1** the appropriate number of times.
3. Your device has an automatic cut-out function which switches it off automatically after 20 minutes of treatment.
4. A treatment program with three different pulse patterns runs during the preset 20-minute treatment period.

Pulse pattern 1:

Pulse frequency 30 - 60 Hz

Pulse width 250 milliseconds

Pulse frequency varies automatically in a 6 second on, 1 second off rhythm

Pulse pattern 2:

Pulse frequency 30 Hz

Pulse width 150 - 250 milliseconds

Pulse frequency varies automatically in a 6 second on, 1 second off rhythm

Pulse pattern 3:

Pulse frequency 30 - 60 Hz


Pulse width 150 - 250 milliseconds

Pulse frequency varies automatically in a 6 second on, 1 second off rhythm


The three pulse patterns run consecutively for 1 minute 20 seconds each. This pattern is repeated 5 times during the 20-minute treatment period. The device's operation is displayed visually in that function display **3** flashes with decreased brilliancy in line with the rhythm of the treatment program.

5. Treatment may be discontinued at any time by pressing – /OFF button **2** repeatedly (depending on the level of intensity) until red function display **3** lights up more brightly again. Now press button **2** one more time and the device is switched off.
6. The adhesion of electrode pad **4** is very strong when used the first few times. Therefore, take great care when removing the electrode pad and device from the skin. Never pull on the device. After use, replace the electrode pad on the protective plastic film for storage.


4.2
Positioning the
Electrode Pad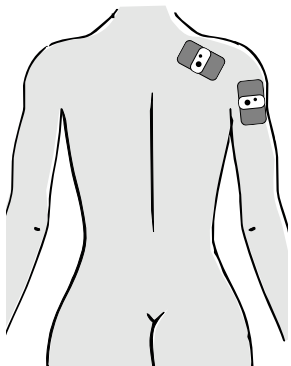


Tension in shoulder
 and arm

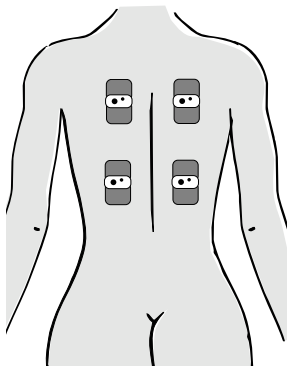
Tension in upper
 and lower back

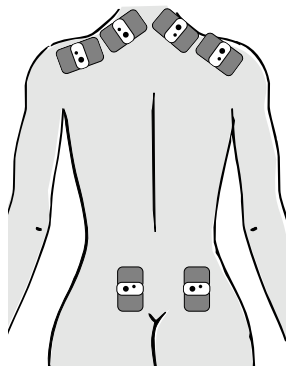
Tension headache
 and tension in nape,
 sciatica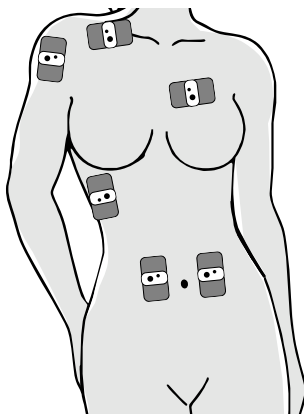

Strains, muscle cramps
 and menstrual pain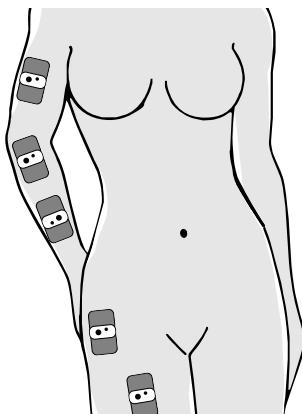

Tennis elbow, joint pains
 and bruising

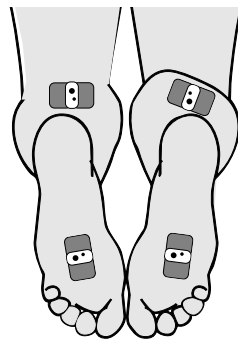Cramp
 and circulation problems

5.1 Trouble- shooting

Please contact your service centre if it is not possible to remedy a fault in the manner described.

| Fault | Cause | Remedy |
|---|--|--|
| LED fails to light up | No battery in the device or battery spent | Replace battery |
| | Battery not inserted correctly | Insert battery again |
| | Device not switched on Insulating film has not been removed | Switch device on Remove insulating film from between battery and device |
| LED lights up weakly | Battery spent | Replace battery |
| Low output from device | Electrode pad dirty | Clean/replace electrode pads |
| | Device not applied properly | Re-apply device |
| LED function normal but no power output | Battery spent | Replace battery |
| | Electrode pad dirty or short-circuited | Check electrode pads and replace if necessary |
| Device switches off prematurely | Battery spent | Replace battery |
| | Electrode pad worn out or dirty | Check electrode pads and replace if necessary |
| | Device not applied properly | Re-apply device |

5.2 Cleaning and maintenance

Electrodes:

- Clean the adhesive surface after use to achieve better adhesion. Moisten the gel with a drop of water and allow it to dry briefly. Then stick the electrode back on the protective film.
- The length of time the electrodes remain usable depends on their storage, skin type and use. Store the electrodes in their storage box in a cool, dry place.

Device:

- The device does not require any special maintenance.
- Remove the battery before cleaning the device.
- Never use aggressive cleaning products or stiff brushes.
- Clean the device with a soft, possibly slightly moistened cloth. Do not allow water to penetrate the device. Do not use the device again until it is completely dry.
- Remove the battery from the device if it is not going to be used for some time.
- Do not expose the device to direct sunlight and protect it against dirt and moisture.
- With the electrode pad on the protective plastic film, store the device in a cool, dry place in the storage box provided on purchase.

5.3 Disposal

This product must not be disposed together with the domestic waste.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Please remove the battery before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop.

Consult your municipal authority or your dealer for information about disposal.

5.4 Technical data

| | |
|-----------------------|---|
| Name and model | : MEDISANA Pain relief plaster Medistim |
| Channel | : Single channel |
| Voltage supply | : 1 lithium coin cell battery (CR 2032) |
| Output voltage | : 0 - 30 V \pm 20% (at 500 Ω) |
| Pulse intensity | : Adjustable from 0 - 60 mA \pm 20% |
| Pulse pattern | : 3 different automatic pulse pattern sequences, each 1 min 20 sec. |
| Pulse width | : 150 - 250 milliseconds (depending on pulse pattern) |
| Autom. cut-out | : After 20 min |
| Operating conditions | : +10 °C to +40 °C, 40 - 90 % max. relative humidity |
| Storage conditions | : - 10 °C to +60 °C, 30 - 95 % max. relative humidity |
| Dimensions | : Approx. 30 mm x 58 mm x 10 mm (base unit) |
| Weight | : Approx. 20 g incl. battery and large electrode |
| Medistim | : Article no. 81700 : EAN-Nr. 4015588817007 |
| Medistim-Duo | : Article no. 81800 : EAN-Nr. 4015588818004 |
| Replacement electrode | : Standard (large) Article no. 81820 : Special (small) Article no. 81810 |

CE 0297

In accordance with our policy of continual product improvement, we reserve the right to technical and design changes without notice.

6.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to send in the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA AG

Itterpark 7-9

D-40724 Hilden

Tel.: +49 (0) 2103 / 2007-60

Fax: +49 (0) 2103 / 2007-626

eMail: info@medisana.de

Internet: www.medisana.de

The service centre address is shown on the last page.

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D&L Marketing Oy
ALERE Electronics AB