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## Thank you very much

Thank you for your confidence in us and congratulations!  
The **PSB** personal scales with body analysis functionality which you have purchased are a top quality product from **MEDISANA**.

This unit is intended for weighing and calculating the proportion of body fat, water content and muscle mass in people.

In order to achieve the desired effect with your **MEDISANA** Personal scales **PSB** with body analysis functionality in the long term, we recommend that you read the following information on its use and maintenance carefully.

### 1.1 Important

**Please read the following safety information carefully before using the device and keep these instructions in a safe place for later use.**

- Any treatment or diet for people who are underweight or overweight requires the qualified advice of a professional (doctor, dietician). The values determined with the scales may be source of useful support.
- This device is not suitable for pregnant women!
- These scales are not suitable for people with pacemakers or other medical implants. The body fat information may be inaccurate in people with diabetes or other medical/physical limitations. The same also applies to very highly trained athletes.

- **Risk of tipping over!**  
Do not place the scale on an uneven surface.  
Do not stand on the corners/edges of the scale.

- **Risk of slipping!**  
Do not stand on the scale with wet feet.  
Do not stand on the scale wearing socks.
- Use the device only according to its intended purpose as specified in the instruction manual.
- The warranty will be invalidated if the device is used for purposes other than those for which it is intended.
- The scale has been manufactured for domestic use. It is not suitable for commercial use in hospitals or other medical institutions.
- Do not use the device if it is not working correctly, if it has been dropped or has fallen into water or has been damaged.

- The scales have a measuring range up to 180 kg, 396 lbs/19 st. Do not overload the scales.
- Do not place the scale on soft or uneven surfaces. The most accurate measurements will be obtained with the scale on a firm, level surface.
- Do not use the scale in a location where extremely high temperatures or humidity may occur.
- Keep the scale away from water.
- Handle the scale with care. The scale must not be subjected to impacts or vibration. Do not drop the scale.
- Do not attempt to dismantle the scale, otherwise the warranty will be invalidated. The scale does not contain any parts that need to be maintained by the user, with the exception of the battery.
- Do not attempt to repair the device yourself in the event of a malfunction since this will invalidate the warranty. Repairs should only be carried out by authorised service centres.
- If the scale is not going to be used for a long period, please remove the batteries.

## 1.2 Scope of Supply and Packaging

Please check first of all that the unit is complete.

The following parts are included as standard:

- 1 **MEDISANA PSB** Personal scales with body analysis functionality
- 2 Type CR2032 3V lithium coin cell batteries
- 1 Instructions for use

Packaging is re-usable or may be recycled. Please dispose of unwanted packaging material in the proper manner. Please contact your dealer if you notice any transport damage on unpacking the unit.

### **WARNING**

**Do not allow children to play with the packaging film.  
Risk of suffocation!**

### 2.1 Performance features

- Top quality safety glass
- Top quality sensor technology
- Age entry from 10 – 99 years
- Height entry from 100 to 220 cm
- Measuring range 5 kg to 180 kg or 11 lbs to 396 lbs/19 st
- 100 g or 0.2 lb graduation
- Switchable between kg and lbs/st
- Body fat analysis 5 - 65 % in 0.1 % steps
- Water content measurement 20 - 72 % in 0.1 % steps
- Muscle mass measurement 20 – 70 % in 0.1 % steps
- Calorie intake/day display in Kcal
- 12 memory locations
- Direct measurement of body weight
- Easy to use
- Clear LCD display
- Includes 2 type CR2032 3V batteries

### 2.2 Why is it important to know how much body fat you have?

The human body is made up amongst other things of water, fat and muscle mass. A fat proportion which is too high or too low may lead to increased health risks. The personal scales with body analysis functionality provide useful support in motivating someone to reach or maintain the best possible body weight by monitoring the proportion of body fat.

### 2.3 How does the scale with body analysis functionality work?

The scale measures using BIA (Bio-electrical Impedance Analysis). When you step onto the metal plates barefoot, extremely weak electrical current is passed into your body. The scale weighs the difference between these signals. The measurements are used in combination with personal data such as age, gender, height and weight to calculate the amount of body fat.

### 2.4 Prerequisites for correct measuring results

#### NOTES

**Excessive drinking, eating, training, medical treatment, the female menstrual cycle etc. affect the measuring results. The body fat, water content and muscle measurements will be incorrect for women during pregnancy.**

**2.4 Prerequisites for correct measuring results**

**Reliable measurements can only be achieved under certain prerequisites:**

- Always measure barefoot.
- The feet must have a good contact with the electrodes on the standing area.
- Stand upright.
- Carry out the measurement approximately 15 minutes after getting up, or after bathing or showering, with towel-dried feet.
- Please measure at the same time of day if possible.
- Do not measure after strenuous physical exercise. Wait at least an hour before measuring.
- There must not be any dirt stuck to the soles of your feet. Erroneous measuring results may occur if the soles of your feet are dry or extremely callused.

The amount of body fat, water content and muscle mass can only be reliably monitored using the measurements if they are taken consistently. Measurements that are taken under varying conditions can return different values.

**2.5 Body fat proportion range in %**

 **NOTES**

**The information in the tables is for orientation purposes only. If you have any questions about your proportion of body fat, please consult your doctor.**

<i>Female</i>	Age	under-weight	normal	over-weight	fat
	15-39	14-20	21-25	26-31	32-38
	40-55	15-21	22-26	27-32	33-38
	56-70	16-22	23-27	28-33	34-38
<i>Male</i>	Age	under-weight	normal	over-weight	fat
	15-39	11-17	18-23	24-29	30-36
	40-55	12-18	19-24	25-30	31-37
	56-70	13-19	20-25	26-31	32-38

**Body fat value analysis:**

In order to analyse your body fat proportion, please compare your value to the values in the body fat table.

### 2.6 Body water proportion range in %

<i>Female</i>	Age	low	normal	high
	10-15	< 57	57-67	> 67
16-30	< 47	47-57	> 57	
31-60	< 42	42-52	> 52	
61-80	< 37	37-47	> 47	

<i>Male</i>	Age	low	normal	high
	10-15	< 58	58-72	> 72
16-30	< 53	53-67	> 67	
31-60	< 47	47-61	> 61	
61-80	< 42	42-56	> 56	

#### Body water value analysis:

In order to analyse your body water value, please compare your value with the values in the body water table.

### 2.7 Average Muscle Mass

The average proportion of muscle mass for women between 15 and 70 years of age is max. 34% and for men 40%. Compare your value with the standard value to analyse your muscle mass.

### 3.1 Insert/replace batteries

Before you start up your digital scale, insert the two provided cell batteries (type CR2032) into the device. To do this, open the battery compartment **5** beneath the device and insert the batteries. Please note polarity (positive pole upwards as shown on the lid of the battery compartment). Close the battery compartment again. Replace the batteries if the battery change symbol "Lo" appears in the display **1** or if nothing appears in the display when the device is switched on.


#### BATTERY SAFETY INFORMATION

- **Keep away from children.**
- **Do not short-circuit.**
- **Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop.**
- **Do not recharge.**
- **Do not throw into fire.**

**3.2**  
**Weight only /**  
**“step on” function**

1. Stand the scales on a firm, even floor. Step onto the scales and stand still.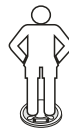

2. The unit switches on automatically and “0.0 kg” appears briefly on the display ❶.

3. Your weight is measured and displayed (in the example shown: 68 kg).

4. Step down from the scales. The display changes to “0.0 kg”. The scales turn off automatically after 36 secs if no further operation is performed.


Alternatively, you may switch the scales on with the ON button ❷, step onto the scales and read off your weight; the weight display flashes twice before the scales switch off automatically.

**3.3**  
**Change the**  
**Unit of Weight**

1. Press the ON button ❷, to switch on the scales. “0.0 kg” appears on the display.


2. Turn the scales over and select the desired unit of weight (kg - st - lb) by pressing the black button.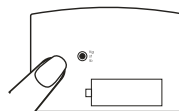


**3.4**  
**Programming**  
**personal data**

The scale can store information for 12 persons: gender, age and height. The scale contains preset values. These values are: **kg, male, 170 cm, 30 years of age**. Place the scale on a stable, firm surface. All four feet must be touching the floor.

1. Press the ON button ❷ to switch on the scales. “0.0 kg” appears on the display.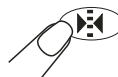

2. Press and hold the ON button ❷ for approx. 3 secs; the memory location setting starts to flash.

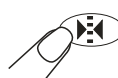3. Select the desired memory location for your user profile (1 - 12) by pressing the UP button **4** and/or the DOWN button **3**.

4. Press the ON button **2** to save the setting. The "Man" symbol starts to flash on the display.

5. Set your sex by selecting the appropriate symbol on the display with the UP button **4** and/or the DOWN button **3**.

6. Press the ON button **2** to save the setting. The preset age on the display starts to flash.

7. Now set your age using the UP button **4** and/or the DOWN button **3**.

8. Press the ON button **2** to save the setting. The preset height on the display starts to flash.

9. Set your height using the UP button **4** and/or the DOWN button **3**.

10. Press the ON button **2** to save the setting.

11. The setting procedure is now complete. The display of the memory location on the display flashes another five times before the unit switches off automatically.

The UP button **4** and/or DOWN button **3** may be pressed before the unit switches off to program further memory locations. Proceed as described above and repeats steps 4 – 10.

The scales will switch off automatically if no button is pressed during the programming sequence for approx. 18 secs.

**3.5 Weighing and measuring body fat, water content and muscle mass**

**⚠ NOTE**

Reliable measurements can only be achieved under certain circumstances, as described in "2.4 Prerequisites for correct measuring results", p. 14 - 15.

Place the scales on a firm, hard surface. All four feet must be in contact with the floor.

1. Press the ON button **2** to switch on the scales. "0.0 kg" appears on the display.



2. Select your personal memory location (1 - 12) by pressing the UP button **4** and/or DOWN button **3**.

3. Press the ON button **2** to confirm the selection. Your personal store data are displayed; then "0.0 kg" appears again on the display.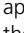

4. Step onto the scales with bare feet and stand still. The scales will switch off automatically to save the batteries if you have not stepped onto the scales within approx. 36 secs. First of all your weight is shown for approx. 8 secs. At the same time, the  symbol flashes which shows that the body fat value, the water content and the muscle mass are being measured.

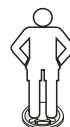
5. The measured values for body fat, water content and muscle mass are shown on the display **1** one after another for approx. 5 secs each followed by the value determined for the daily calorie intake.


The display sequence of the measured values is repeated. Then the scales switch off automatically. Step down from the scales.

**Evaluation of the proportion of body fat**

<b>Explanation of symbols:</b>	Low	
	Normal	
	Too high	
	Much too high	

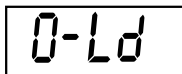**3.6  
Error messages**


The scale is overloaded.

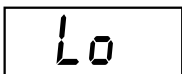The battery is low and must be replaced.

Error during measurement.

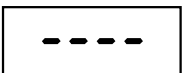Data not saved.

**4.1  
Cleaning and  
maintenance**

- Remove the batteries before cleaning the unit.
- Never use aggressive cleaning agents or stiff brushes.
- Clean the scale using a soft cloth lightly moistened with a mild soapy solution. Never use abrasive cleaning agents or alcohol.
- Do not allow any water to get into the unit. Do not use the unit again until it is completely dry.
- Remove the batteries from the unit if it will be unused for a longish period. Otherwise there is a risk that the batteries might leak.
- Do not expose the unit to direct sunlight; protect it against dirt and moisture.

## 4.2 Disposal

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Please remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop.

Consult your municipal authority or your dealer for information about disposal.

## 4.3 Technical data

Name and model	:	<b>MEDISANA PSB</b> Personal scales with body analysis functionality
Dimensions (L x W x H)	:	approx. 28.4 cm x 28.7 cm x 4 cm
Weight	:	approx. 1.9 kg
Voltage supply	:	6 V= , 2 x 3 V lithium coin cell batteries type CR2032
Display system	:	Digital display
Memory	:	for 12 persons
Measuring range	:	5 kg to 180 kg or 11 lbs to 396 lbs/19 st
Body fat analysis	:	in steps of 0.1 %
Body water	:	in steps of 0.1 %
Muscle mass	:	in steps of 0.1 %
Graduation	:	100 g or 0.2 lb
Autom. switch-off	:	after approx. 36 sec.
Item number	:	40410
EAN number	:	4015588 40410 8

**We reserve the right to make technical and design changes in the course of continuous product improvement.**

### 5.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to send in the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

#### **MEDISANA AG**

Itterpark 7-9

40724 Hilden

Tel.: +49 (0) 2103 / 2007-60

Fax: +49 (0) 2103 / 2007-626

eMail: [info@medisana.de](mailto:info@medisana.de)

Internet: [www.medisana.de](http://www.medisana.de)

The service centre address is shown on the last page.

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D&L Marketing Oy
ALERE Electronics AB