MEDISANA®

SMS/SML

- DE** Saug-Sicherheitshaltegriff
GB Suction Mount Safety Bar
FR Poignée de sécurité
IT Maniglia di sicurezza a ventose
ES Asidero de seguridad con ventosas
PT Pega de segurança com ventosas
NL Zuig-veiligheidsgreep
FI Imukuppi-tukikahva
SE Stödhandtag med sugfunktion
GR Αναρροφητική ασφαλής λαβή

Art. 48920 SMS
Art. 48925 SML

Gebrauchsanweisung
Manual
Mode d'emploi
Istruzioni per l'uso
Instrucciones de manejo

Bitte sorgfältig lesen!
Please read carefully!
A lire attentivement s.v.p.!
Da leggere con attenzione!
¡Por favor lea con cuidado!

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IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

Lot number

Manufacturer

GB 1 Safety Information

MEDISANA®

1.1 Safety Information

- Only use this unit according to the intended purpose and as directed in the instructions for use. The device is not intended for commercial use.
- Check the suction cup **3** prior to every use for damage and that it is functioning correctly! Do not use the safety bar if you have any doubts about its integrity!
- If there are any defects, do not repair the device yourself and do not continue to use it! Consult your specialist dealer and only have repairs carried out by authorized service personnel.
- You may only clean the unit yourself.
- The safety bar assists and supports natural movements when taking a shower or a bath.

WARNING

This safety bar can only have a supportive effect! It is not suitable for holding your entire body weight!

- The maximum weight capacity of the Suction Mount Safety Bar **SMS/SML** amounts to:
 - approx. 75 kg vertical weight capacity
 - approx. 50 kg lateral weight capacity
 - approx. 40 kg angled weight capacity

WARNING

Please look at the suction force safety indicators before each use **2! When you move the suction power lever **1**, this must change to the colour "GREEN"!**

A Safety Bar which is securely fixed at first can, depending on the surface, gradually get loose! Therefore, prior to each use, check the suction force safety indicators and/or that the bar is securely affixed.

- This product is only suitable for use on **absolutely smooth**, non-porous surfaces: tiles, glass, ceramic or fibreglass. It may not be used on shelves, wood, painted or paper-covered surfaces!
- The surfaces (e.g. tiles) must be firmly attached to the substrate.
- Do not affix the Safety Bar to any moving surfaces!
- Do not expose the device to direct sunlight.
- Do not use any sharp or pointed objects to remove the Safety Bar (**Risk of damage!**).
- Leave the suction force levers **1** in the raised position when you are not using the Safety Bar.

Thank you very much

Thank you for your confidence in us and congratulations on your purchase!
With the Suction Mount Safety Bar **SMS/SML** you have purchased a **MEDISANA** quality product.

The Suction Mount Safety Bar **SMS/SML** holds in place by creating a powerful suction (vacuum) under the suction cups. This dispenses with the need for tools, screws or rawlplugs.

To derive the most success and to enjoy your **MEDISANA** Suction Mount Safety Bar **SMS/SML** for a very long time, we recommend that you read the following information on the use and care of the device carefully.

2.1 Items supplied and packaging

Please check first of all that the unit is complete and is not damaged in any way. If in doubt, do not use the appliance. Send it to a service point.

The following parts are included as standard:

- 1 Suction Mount Safety Bar **SMS** or **SML**
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.

If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING

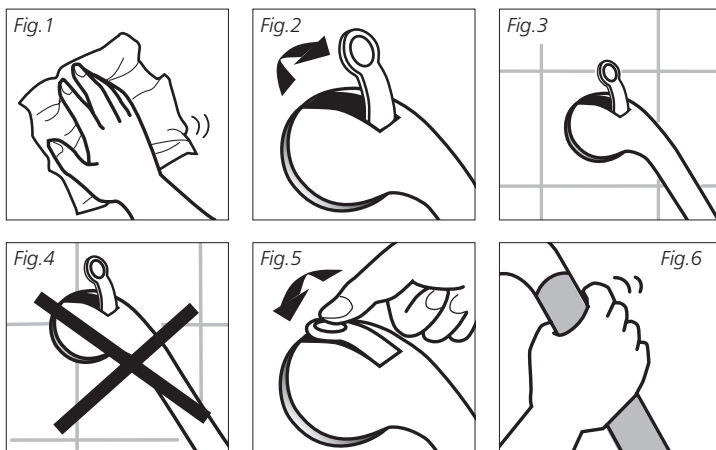
Please ensure that polybag packing is kept away from reach of children! Risk of suffocation!

GB 2 Operating

MEDISANA®

**2.2
Attaching the
Safety Bar**

1. Clean the **absolutely smooth** suction area using a cleaning agent (fig. 1) and let the area dry completely. Check the suction cup **3** prior to every use for damage and that it is functioning correctly! Do not use the safety bar if you have any doubts about its integrity!

2. Position both suction force levers **1** facing upright (fig. 2). Ensure to place the Safety Bar completely and directly on the **absolutely smooth** suction area (fig. 3). Do not affix it to grout or cavities, e.g. tile grouting (fig. 4). **Please observe the safety instructions!**
3. Press the levers downwards one at a time while firmly holding the Safety Bar at the opposite end (fig. 5). The suction force safety indicators **2** are green. The suction is easy to feel (fig. 6).

**WARNING**

A Safety Bar which is securely fixed at first can, depending on the surface, gradually get loose! Therefore, prior to each use, check the suction force safety indicators and/or that the bar is securely affixed.

**2.3
Removing the
Safety Bar**

1. Position both suction force levers **1** facing upright.
2. If suction remains, gently lift one edge of the suction cup **3** with a finger.
3. Do not use any sharp or pointed objects to remove the Safety Bar (**Risk of damage!**).
4. Leave the levers **1** in the **raised position** until the next time you use the Safety Bar.

2.4 Suction force safety indicator

The **MEDISANA** Suction Mount Safety Bar **SMS/SML** is equipped with a amended suction force safety indicator **2**. If this shows the colour "GREEN", this means that the safety bar has been correctly attached. If this shows the colour "RED", this means that the safety bar has not been attached correctly and/or is not secure and must be reattached.

3 Miscellaneous

3.1 Cleaning and maintenance

- Clean the device with a damp, soft cloth. You can use mild household or bathroom detergent diluted with water.
- Do not use abrasive cleaning agents. They could damage the surface.
- Do not use any sharp or pointed objects to remove the Safety Bar from the suction area (**Risk of damage!**).
- Leave the suction force lever **1** in the **raised position** when you are not using the Safety Bar.
- Dispose of a device which can no longer be used in an environmentally sound manner. If in doubt please consult your local authorities or your retailer.

3.2 Specifications

Name and model	: MEDISANA Suction Mount Safety Bar SMS/SML
Maximum weight	: – approx. 75 kg vertical weight capacity – approx. 50 kg lateral weight capacity – approx. 40 kg angled weight capacity
Dimensions l x w x h	: approx. 350 x 93 x 95 mm SMS approx. 500 x 93 x 90 mm SML
Weight	: approx. 600 g SMS approx. 550 g SML
Article no.	: 48920 SMS 48925 SML
EAN-Number	: 40 15588 48920 4 SMS 40 15588 48925 9 SML

In accordance with our policy of continual product improvement, we reserve the right for technical and design changes without notice.

4.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to send in the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

 **MEDISANA AG**

Itterpark 7-9
40724 Hilden
Germany
Tel.: 0 21 03 - 20 07-60
Fax: 0 21 03 - 20 07-626
eMail: info@medisana.de
Internet: www.medisana.de

The service centre address is shown on the last page.

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ALERE Electronics AB