## TENS-Schmerztherapiegerät TDD

### TENS Pain Therapy TDD

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## GB Display:

- 1 Therapy time setting
- 2 Intensity channel 1
- 3 Operation mode setting/program selection
- 4 Pulse width in µsec
- 5 Frequency in Hz
- 6 Connection socket channel 1
- 7 Connection socket channel 2
- 8 Display current selection
- 9 Intensity channel 2
- 10 Battery display

## Keys:

- A Timer = therapy time setting
- B Wave = current selection
- C Ch1 Intensity = increase intensity, channel 1
- D Ch1 Intensity = reduce intensity, channel 1
- E Frequency = frequency setting
- F On/Off = on/off key
- G P. Width = pulse width setting
- H Ch2 Intensity = increase intensity, channel 2
- J Ch2 Intensity = reduce intensity, channel 2
- K Mode = operation mode setting/program selection

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### 1.1 Thank you very much

Thank you for your confidence in us and congratulations!

The **TDD** pain therapy unit that you have purchased is a top quality product from **MEDISANA**.

In order to achieve the desired effect with your **TDD pain therapy unit** in the long term, we recommend that you read the following information on its use and maintenance carefully.

### 1.2 Important

**Please read the following safety information carefully before using the device and keep these instructions in a safe place for later use.**

- Use the device only according to its intended purpose as laid down in the instruction manual.
- Do not use accessories that are not recommended by the manufacturer.
- Do not put any object into one of the outlets. Avoid the contact with pointed or sharp objects. When using it for other purposes, the warranty expires.
- If the device is used from, on or near children or people who are handi-capped and can only take limited use of the device, this should always be monitored. Medical devices are no toy!
- In case of a malfunction, do not try to repair the device yourself. Not only does the warranty expire, but serious danger can result (fire, electric shock, injuries). Only authorised service offices should carry out a repair.
- Never use the device while taking a bath or shower. Should liquids penetrate the device, immediately remove the storage battery and avoid further application. Contact your specialist shop or your servicecentre.
- Storage batteries have to be disposed of in special collecting containers or a specialist shop. Never throw them into the household garbage.

### 1.3 Important advice - for your health's sake

- In case of diabetes or other diseases you should consult your family doctor before applying the device.
- Pregnant women should observe the necessary precautions and their individual endurance; if necessary, contact your doctor.
- Do not treat any parts of the body that have swellings, burns, inflammation, peeling skin, wounds or other sensitive points.
- The treatment should be pleasant. If you feel pain or feel the application unpleasant, stop it and consult your doctor.

1.4

**Caution!**  
**Special features**  
**of the**  
**MEDISANA TDD**

- Therapys with the **MEDISANA TDD** do not replace medical diagnosis or treatment. Ask you doctor for each kind of pain or disease, before using the device.  
A TENS therapy should only be done according to instructions of your doctor or the recommendations of application in the TENS therapy book.
- Do not use the device if you have a pacemaker or other implants or prosthetic materials in your body.
- If irregularities of the heart or heart diseases are known (e.g. arrhythmia or myocardial defects), you should not apply the device.
- Persons, whose sensitivity is disturbed or impaired may only apply the device according to the instructions of a physician.
- Do not apply the device near the larynx.
- Do not apply the **MEDISANA TDD** during the operation of a machine or while driving a car.
- If the device shows obvious damages at the housing or the LCD display, send it to your specialist shop or directly to **MEDISANA** for repair.
- Do not use damaged or defect lines to the electrode; there is danger of electric shock.
- When using the **MEDISANA TDD** device for commercial use, a safety control has to be carried out every 2 years at an authorized service office or directly at **MEDISANA** (MPBetreibV = operator directive).

**Informations about the electrodes:**

- Do not place the electrodes to open skin parts.
- Take off the electrodes before taking a bath or swimming.
- Use separate electrodes for each user.
- In case of skin irritation, stop the treatment and consult a physician.

2.1

**Items supplied**  
**and packaging**

Please ensure that the package contents are complete.  
These are as follows:

- 1 TENS unit with belt clip, 4 electrodes,
- 2 connecting cables, 2 batteries and 1 storage box
- 1 TENS therapy booklet, 1 instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

**WARNING**

**To avoid risk of suffocation, keep packaging film away from children!**

## 2.2 Valuable information about stimulation current

Stimulation current can be used:

1. to fight pain (TENS = Transcutaneous Electrical Nerve Stimulation)
2. to build up the muscles  
(EMS = Electrical Muscle Stimulation)
3. as electro-acupuncture.

### 2.2.1 Fighting pain (TENS)

TENS is understood as electrical nerve stimulation through the skin.

Pain is a natural and important warning signal of our body telling us that "something is wrong". If the reason for pain is diagnosed, but cannot or can only partially removed, a long-lasting, persistent pain evolves (= chronic pain). Now, this is in its former function as warning signal useless and can largely affect the life of the person suffering, maybe even cause after-effects like tensions, depression or less joy of living.

Chronic pain can be treated with different methods: e.g. with medicaments or operations, but also with alternative, methods like acupuncture or TENS that are free of side-effects.

On the one hand, the TENS treatment can block pain signals by interrupting the transmission of impulses through the skin to the painful parts by means of electric impulses. The signal "pain" is not given on to the brain and thus not felt as pain.

On the other hand, special TENS applications activate the body's control mechanisms. Gentle electric currents make the body to release so-called "endorphins" that function as pain reducer. The pain is thus stopped, eased or removed by the body itself.

This electrical stimulation cannot remove the causes or cure the disease, but change the painful feeling. Painless conditions or conditions freed of pain can help the chronically suffering person to make new, positive experiences, from which he or she can gain power and energy.

### 2.2.2 Building up muscles (EMS)

With different EMS applications, muscles in the body can be stimulated, which are e.g. slackened after an operation, trained one-sided or unsufficiently or which are tensioned. In the process, electric pulses cause slight muscle contractions that can stimulate the muscular system "passively" (= without the muscles self-acting).

### 2.2.3 Electro- acupuncture

Acupuncture is an old chinese healing method basing on the fact that there are points in the body (= acupuncture points), on the stimulation of which has direct effects (e.g. animation, pain relief, increase of the energy flow etc.) on certain body zones or body organs. This stimulation can be done with needles, massage, pressing (= acupressure) or with electric pulses, e.g. TENS.

Detailed information can be found in the relevant specialist literature.

## 2.3 Inserting/changing batteries

Slide the cover of the battery case at the back of the device away in arrow direction. Put the enclosed batteries according to the symbols (+/-) into the base of the battery case. Close the battery case and push the cover towards the device until it snaps into position.

When changing the batteries, only use 1.5 V batteries type AA or LR6.

**BATTERY SAFETY INFORMATION**

- **Keep away from children!**
- **Do not short-circuit!**
- **Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop!**
- **Do not recharge!**
- **Do not throw into fire!**

## 2.4 Cover

For setting the **MEDISANA TDD**, you slide the cover below the LCD display down. Now all operation keys of the device are accessible. You can set the device according to your very special needs or select the preset fixed programs. In the programs, frequency, pulse width, current selection and time are pre-defined so that you only need to adjust the intensity.

## 2.5 Cable connection

At the upper side of the device there are two sockets for the connection of the electrode line. The left socket **6** is for channel 1, the right socket **7** is for channel 2. Connect at least one of the lines by putting the cable tightly into the socket. At the other end of the cable, you fasten an electrode. Place the electrode(s) according to your area of pain (see enclosed TENS booklet or/and observe your doctor's instructions).

## 3.1 Application fields

The **MEDISANA TDD** can be used for all kinds of pain or for building up muscles, for long-term therapy as well as for acute treatment actually.

**NOTE**

Observe your doctor's precise instructions and let him show you where to apply the electrodes. For a successful therapy, the correct application of the electrodes is an important factor. Carefully write down the settings of frequency, pulse width and current selection your doctor recommended. For this, you best take your **MEDISANA TDD** to the doctor so that he can show you the settings.

The accompanying TENS therapy booklet or special textbooks provide you with detailed information concerning the subject "pain, transcutaneous electrical nerve stimulation (TENS)", concerning indications and the exact electrode application for an effective success of treatment as well as recommendations for the programs to use and notes on the settings of frequency, pulse width, time of therapy and current selection.

### 3.2 Applying the electrodes

Before placing the electrodes, remove residual skin cream or ointment. The skin has to be free of fat.

Do not place the electrodes on injured or inflamed skin (wounds, pimples, skin rashes, reddenings etc.). Always place the electrodes according to the doctor's instructions or the TENS therapy booklet, before you switch on the device.

- In most cases, the electrodes are placed near the area of pain or directly on the aching spot.
- For the pain and acupuncture program of the **MEDISANA TDD** device, you don't have to pay attention to the polarity of the electrodes.
- For the muscle stimulating program, observe your doctor's instructions concerning the polarity or the explanation in the enclosed TENS therapy booklet.
- If you are not sure how to place the electrodes, exchange the connectors red and white at the electrodes after half of the therapy time.
- If you don't find your case in the TENS therapy booklet, feel for the area of pain, place one electrode directly on it and the second one to the periphery of the area of pain or, on arms and legs directly opposite. In the head area, place the electrodes only on the regions specified by the doctor or the enclosed TENS therapy booklet and perform therapy with very low current intensity.
- After application, stick the electrodes back to the carrier paper. This way, you prolong the electrodes life. Electrodes that do not stick anymore can be cleaned under cold running water. After cleaning, leave the electrodes for 24 hours on the carrier paper so that the adhesive strength is restored.

Only use the original **MEDISANA** electrodes that are enclosed to the device.


Use separate electrodes for each user. They can be reordered as electrode set in different sizes at **MEDISANA**.

#### CAUTION

**Make sure that the electrodes cannot fall off during therapy. This can cause an unintentional electric shock. Switch off the device immediately, if an electrode comes off the skin during treatment.**

#### Therapy examples:

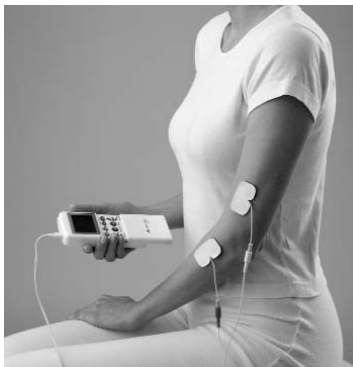Pain in the elbow

Pain in the shoulder and pelvis area

Tensions in the neck and shoulder area

You find more examples for application in the TENS therapy booklet or other specialized text books.

### 3.3 Therapy with a preset program

#### programs 1-3

Open the key cover and press the key **ON/OFF**. On the LCD display, the device's factory setting appears:

Frequency 2Hz, pulse width 50 µsec, pulse form biphasic and operation modus constant, timer is not activated.

The marking CH1 **2** for channel 1 CH2 **9** for channel 2 also appear. For selecting the therapy program, press the key M (K), until the desired therapy program is displayed.

#### CAUTION

**The programs 1-3 start as soon as you have selected them.**

For therapy, you increase the intensity with the key **C** (left cable connection) or **H** (right cable connection). Press the key repeatedly, until you feel that the device delivers current.

You can read the intensity of the set current off the height of the bar above CH 1. If you have set the intensity too high, reduce it with the key **D** or **J**. In the course of the programm, always change the intensity when the current strength becomes unpleasant or the tingling lessens.

After the end of the therapy time (program 1-3 automatically preset to 30 minutes each), the device switches off automatically

### 3.4 Therapy according to individual presets

Switch on the device with the key **ON/OFF**. The device is now set to the factory setting:

Frequency 2Hz, pulse width 50 µsec, pulse form bi-phasic and operation mode constant, timer not activated.

The designations CH1 **2** for channel 1 and CH2 **9** for channel 2 are shown this way as well. Set the parameters exactly according to the instructions of your doctor and adjust the intensity with the keys **C**, **D** and **H**, **J**.

If you have not set the therapy time (**A/1**), make sure not to have the device switched on longer than 30 minutes.

If the timer is set, the device switches off automatically after the preset time.

**NOTE**

Some of the settings in the different operating modes may have changed, even after treatment has already started (see page 33).

Close the key cover to prevent your device's individual setting from being accidentally changed.

In the course of the therapy application, always change the intensity when the strength of current becomes unpleasant or the tingling lessens.

**3.5****Explanation of the key functions****A Timer = therapy time setting**

After switching on the device (F), no time is set at the beginning, i.e. you fix the therapy length yourself and complete the therapy either by switching the device off (F) or by reducing the intensity to zero (D and/or J). With the timer key (A), you set the therapy time that is shown in minutes ① on the display at the right side of the clock symbol.

If you press the key (A) once, the note "20 min" is displayed. The device now switches off automatically after beginning of the treatment, that means after increasing the intensity (C und/oder H), after 20 minutes.

If you press the key a second time, the display changes to "40 min". The therapy time now is 40 minutes.

Pressing the key once more switches off the time setting.

This key function is not active, if you have set the programs 1, 2 or 3 with the mode key (K). These programs have a fixed duration of 30 minutes and the device switches off automatically after this time.

**B Wave = current selection**

The device can generate different current selections for the particular therapies. They are distinguished between bi-phasic and mono-phasic pulses. In the case of bi-phasic pulses, the current flows back and forth between the electrodes. In the case of mono-phasic pulses, the current only flows in one direction.

After switching on, the device is in the factory setting bi-phasic (8, see diagram 1).

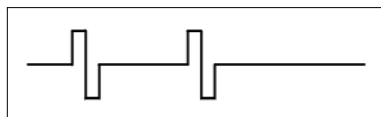
diagram 1

This setting is preferably used with high frequencies (also see "E frequency setting") for pain relief. The positive (rectangular curve upwards) and negative (rectangular curve downwards) parts follow each other in quick succession without break.

Press the key (B) once, and the display changes to the second bi-phasic pulse. The positive and negative part of the pulse are here separated by one span (see diagram 2).

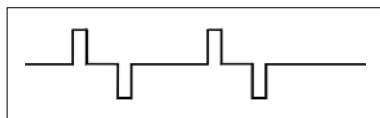
diagram 2

This pulse form can be used for pain relief with high frequencies (from 60 Hz on) as well as for muscle stimulation with low frequencies (2 to 40 Hz).

Press the key once again, and the display changes to the mono-phasic pulse (see diagram 3).

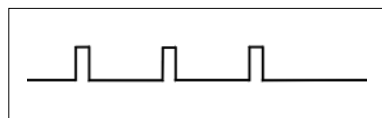
diagram 3

This pulse is exclusively used for muscle stimulation with low frequencies (up to 40 Hz).

Pressing the button once more brings you back to the factory setting to the first bi-phasic pulse.

### **C** Ch1 Intensity = higher intensity, channel 1

The intensity of each channel can be adjusted separately. With the key C you increase the current strength at channel 1, i.e. the strength of the current signal sent "upwards and downwards" (see diagram 4):

diagram 4

From 0 to 20 maximum you can increase with each depression of the key. This is shown in the display **2** as bar.

### **!** CAUTION

**Before you increase the intensity, always connect yourself to the device by putting the cable in socket 6 and applying the electrodes (also see "Applying the electrodes").**

Always start with the lowest step and increase slowly. To increase the intensity, press the key once and observe your sensitivity of current. If you don't feel anything, press the key once again and carefully check your feeling, until you feel a tingling or slight twitches under or around the electrodes.

With each depression of key the bar becomes more and more visible by one step.

For pain relief only increase the intensity, until you feel the application pleasant. Under no circumstances the current may be painful.

For muscle relaxation, only increase the intensity until you feel slight twitches under or around the electrodes. Under no circumstances the muscles may painfully contract.

Stop the application if you feel it unpleasant or if pain occurs.

#### NOTE

After about 2 minutes from beginning of the therapy the feeling of current can change. Increase the intensity step by step until you sense again a pleasant feeling.

#### **D** Ch1 Intensity = higher intensity, channel 1

With this key (D) you reduce the intensity at channel 1 by pressing the key once or several times (the bar becomes lower, display ②).

A current reduction always can be useful when the current selection changes in the programs (also see "K operation mode setting/program selection") or the application is felt unpleasant.

#### **E** Frequency setting

This key is used to adjust the frequency of the therapy current, i.e. the number of pulses that are emitted whilst current is being provided (see diagram 5).

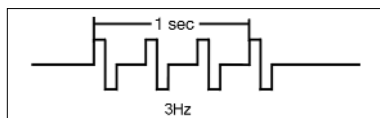
diagram 5

2 Hz to 40 Hz are mainly used for muscle relaxation.

60 Hz to 200 Hz are used for pain relief.

#### CAUTION

Do not change suddenly from 200 Hz to 2 Hz, while you have set the intensity, thus the treatment still runs. This can cause painful twitches.

**F On/Off = on/off key**

With this key you can switch this device on and off (now own display symbol is shown, the factory setting appears in the display when the device is switched on).

**IMPORTANT**

**This function is always active, if you have set the treatment yourself as well as with the therapy with programs.**

You can immediately interrupt each therapy with this key.

If the device does not switch off when pressing the key **F**, the batteries are empty or the device could be defect. Therefore, first exchange the batteries (also see "Changing batteries") and try again.

**G Pulse Width = pulse width setting**

With this key you adjust the pulse width of the individual current pulses, i.e. the length during which the current is sent (see diagram 6).

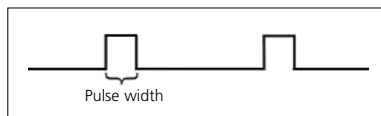
diagram 6

After switching on the device, it is in the factory setting of 50  $\mu\text{sec}$ . By pressing the key you can adjust the pulse width by steps (50-100-150-200-250 $\mu\text{sec}$ ).

Low pulse widths (up to 100 $\mu\text{s}$ ) are preferably used for application in the face or for pain relief.

Higher pulse widths (from 150  $\mu\text{s}$  on) are used for muscle relaxation, especially at the trunk, at arms and legs.

**H Ch2 Intensity = intensity higher, channel 2**

Description of the key function see **C**.

**J Ch2 Intensity = intensity reduction, channel 2**

Description of the key function see **D**.

### **K** Operation model/program selection

With this key you can select the eight preset modes **3**. According to mode, the keys **A**, **B**, **E**, and **G** are activated or deactivated (see table " Available functions - overall view"). With each depression of the key (**K**), you switch to the next mode/program.

After switching on the device, it is in the factory setting, the mode CONST and shows the following values: 2 Hz, 50 µs, 1. bi-phasic current selection.

#### **CONST**

In this mode you can set all values for frequency (**E**), pulse width (**G**) and current selection (**B**) by yourself and constantly keep them.

#### **BURST**

This mode generates pulse packets of the preset frequency that are switches on and off twice a second. The pulse width is adjustable, you can select the current selection as well.

#### **MODUL I**

In this mode, the frequency of the set pulses (pulse width and current selection selectable) is step by step increased from the lowest frequency (2 Hz) to the highest (200 Hz) frequency. The frequency is increased every 10 seconds by one step. After 90 seconds, the device starts again with the lowest frequency.

#### **MODUL II**

In this mode, the pulse width is changed from the highest to the lowest value and back. This process is repeated every six seconds.

With the frequency setting E you can set the frequency of the pulse packets.

#### **PROG A**


The intensity value for the current set by you is lowered to 70% and raised in form of a ramp within 10 seconds. This raising and lowering is not visible in the display. You only feel it as rising and lowering tingling. The settings for current selection, frequency and pulse width are selectable and are not influenced.

**PROG 1**

This program has especially been designed for pain relief. It takes 30 minutes. During this therapy, you will feel five different forms of pulsating and kneading flows (details see table 1).

Frequency (Hz)	Pulse Width (µs)	Duration (min)
200	250	5
120	200	3
80	150	4
200	250	6
80	150	2
120	200	3
200	250	5
80	150	2
		30

**Pulse Shape: Monophasic rectangular**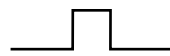

You cannot change frequency, current selection and pulse width while the program is running. However, the intensity is adjustable, and you should correct the intensity, if you feel the treatment too intense or too low (especially after alternation of the flows while the program is running, an adjustment of the intensity to your feeling is recommended).


**PROG 2**

This program has been conceived for a TENS application similar to acupuncture (= acupuncture like TENS = apl-TENS).

It takes 30 minutes during which five different pulsating flows alternate (details see table 2).

Frequency (Hz)	Pulse Width (µs)	Duration (min)
2	100	7
5	100	7
10	100	7
5	100	7
2	100	2
		30

**Pulse Shape: Biphasic**

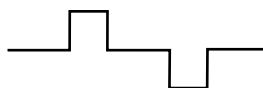You cannot change frequency, current selection and pulse width while the program is running. However, the intensity is adjustable, and you should correct the intensity, if you feel the treatment too intense or too low (especially after alternation of the flows while the program is running, an adjustment of the intensity to your feeling is recommended).

**PROG 3**

This program should be used especially for muscle stimulation. It takes 30 minutes. The program consists of two loops of kneading pulses that are passed through during the whole therapy time. (Details see table 3).

Number	Pulse Width ( $\mu\text{s}$ )	Duration (min)
Loop 1 (x 3)		
1	200 $\Rightarrow$ 2	15
2	Stop	5
3	200 $\Rightarrow$ 2	15
4	Stop	5
5	200 $\Rightarrow$ 2	15
6	Stop	5
		60 x 3 = 180
Loop 2 (x 2)		
7	20	15
8	Stop	7
9	40	15
10	Stop	7
11	60	15
12	Stop	7
13	80	15
14	Stop	7
		88 x 2 = 176

**Pulse Shape: Monophasic rectangular  
with the set pulse width of 250  $\mu\text{s}$**

Program 3 is a combination of 2 loops. The program starts with Loop 1 (number 1 - 6), which is repeated 3 times, lasting for 180 seconds.

After that, Loop 2 (number 7 - 14) is started. It is repeated twice, lasting for 176 seconds.

After Loop 2 is repeated twice, Loop 1 starts again. This process continues on for 30 minutes.

You cannot change frequency, current selection and pulse width while the program is running. However, the intensity is adjustable, and you should correct the intensity, if you feel the treatment too intense or too low (especially after alternation of the flows while the program is running, an adjustment of the intensity to your feeling is recommended).

**NOTE: PROG 1-3**

Each of these programs starts (the set time of 30 minutes starts running), as soon as you have selected it with the key K, whether you have activated the intensity or not. Current pulses are only send, when you increase the intensity. To stop the program, switch off the **MEDISANA TDD** (key F).

**Available functions of the various modes - overall view:**

Modus	Timer (A)	Wave (B)	Frequency (E)	P.Width (G)
CONST	Yes	Yes	Yes	Yes
BURST	Yes	Yes	No	Yes
MODUL I	Yes	Yes	No	Yes
MODUL II	Yes	Yes	Yes	No
PROG A	Yes	Yes	Yes	Yes
PROG 1	No	No	No	No
PROG 2	No	No	No	No
PROG 3	No	No	No	No

**⚠ CAUTION**

**The subjective feeling of current can be changed through alternating frequencies, pulse widths or current selections. Adjust the intensity (keys C, D and H, J), as soon as you feel the application to be unpleasant or the pleasant tingling cannot be felt for a longer period of time.**

**3.6****Description of the symbols in the display****1 Therapy time**

At the right side of the clock symbol, the remaining therapy time is given in minutes. The display changes by pressing the key A from no indication (i.e. you fix the therapy time without indication in the display) to 20 min to 40 min and back. At the moment the intensity in a channel is increased, the therapy starts (Caution except for the programs 1 -3, also see "K operation mode/program selection") and the time is counted down and shown. You can immediately stop each therapy with the key F.

**2 Intensity channel**

By pressing the key C, horizontal lines appear above this writing, showing the set intensity. No bar indicates that no current is activated (Caution except for the programs 1 -3, also see "K operation mode/program selection NOTE"). If the bars are visible up to the upper display edge, the device delivers the maximum current, it is set to maximum intensity.

**3 Operation mode**

In this indication part, the operation modes are displayed: CONST, BURST, Modul I or MODUL II, PROG A, 1, 2 or 3. For more detailed description of the operation modes see "K operation mode/program selection".

**4 Pulse width**

The value of pulse width indicates how long the current for each pulse part is sent with the set intensity strength (also see diagram 6). The value is given in  $\mu\text{s}$  (= microseconds = 1/1.000.000 sec). The smaller the value the shorter the sending time of the respective pulse part.

**5 Frequency**

The frequency shows how many pulses per second are being generated.


The frequency is given in Hz (= 1 pulse unit/second). The higher the frequency value the more often current pulses are sent in one second.

**⚠ CAUTION**

**The respective setting of the pulse width and frequency can result in a feeling of the current as "stronger" or "lower", even if the intensity, i.e. the current strength, is not changed. Adjust - according to the goal of therapy (also see the enclosed TENS therapy booklet) - the values so that the application is always pleasant.**

**8 Current selection**

This display is set with the key **B** (also see "B Wave/current selection"). The two bi-phasic and the mono-phasic current selections are displayed as follows:

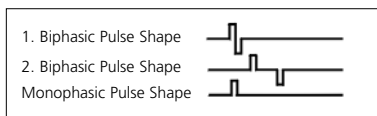**9 Intensity channel 2**

Description of the display see 2.

**10 Battery display**

The two battery symbols start to blink, when the batteries are almost empty. Exchange the batteries, as described in "Inserting/changing batteries". If you use the device frequently, it is recommended to use rechargeable storage batteries. The battery symbols, however, will then start after a shorter period of time, as the storage batteries have a lower cell voltage than alkali-manganese cells.

### 3.7 Duration of treatment

#### 1. Total duration of use

A total time (duration) of use is automatically recorded. When you wish to see the record, press the "M" and "⊖" key together then LCD display will show the number, e.g. 085, this means the unit was used 85 minutes in total after the previous recorded was cleared.

#### 2. Total number of times of use

A total number of times is recorded automatically. When you wish to see the record, press the "M" key and "⏮" key together, the LCD display shows the number, e.g. 009, this means the unit was used 9 times after the previous record was cleared.

#### 3. Clear the above records

How to clear to previous records:

Press the "M" and "μS" key together, LCD display will show  $\frac{000}{000}$ .

#### In simple:

1. "M" + "⊖": total duration of use
2. "M" + "⏮": total number of times of use
3. "M" + "μS": clear the above records

### 4.1 Frequent questions and their answers

#### Can something happen to me, due to the current?

No, because the current strength of the electric pulses is so low that it remains completely safe even with longer applications. However, the intensity should be adjusted to the individual sensitivity.

Therefore, start for each new application with the lowest intensity level and increase it slowly according to the individual feeling (also see: "Change of intensity"). The application should never be unpleasant.

#### I have no experience with stimulation current therapy. How can I apply the device without problems?

The **MEDISANA TDD** has been particularly developed to be easily used by everyone. Even without special knowledge of the subject "stimulation current therapy" you can treat yourself safely and easily with the help of the enclosed TENS therapy booklet and the preset programs 1-3.

#### How often can I use the electrodes?

According to the condition of your skin and when observing the application instructions you can use the electrodes up to 100 times.


#### Can I use other electrodes as well?

Yes, but you should only use original **MEDISANA** electrodes. You can also use duration electrodes with gel (also see "Last but not least, Accessories")

### 4.2 Cleaning and maintenance

- Remove the batteries before you clean the unit.
- Never use strong cleaning agents or stiff brushes.
- Clean the unit with a soft cloth which has been lightly moistened with a mild soap. Never dip the unit into water for cleaning and ensure that no water gets into the unit at any time. Only use the unit again, when it is fully dry.
- If there are separate heads for the unit, then clean them as necessary with a small soft brush. First allow the heads to dry off completely before you use them again on the unit.
- Please clean the electrodes as described in the instruction leaflet.

### 4.3 Disposal

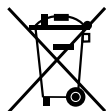This product must not be disposed together with the domestic waste.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Please remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop.

Consult your municipal authority or your dealer for information about disposal.

**4.4  
Accessories  
MEDISANA TDD**

*Ask in your specialist shop or directly at our hotline.*

2 patient cords	item number: 87030
TENS therapy booklet	item number: 86290
4 self-adhesive multiple electrodes, 30 x 50 mm	item number: 87080
2 self-adhesive multiple electrodes, 50 x 90 mm	item number: 87100
4 duration electrodes for the use with electrode gel, 40 x 40 mm	item number: 13586
Electrode gel for duration electrodes, 50 g tube	item number: 12943
2 patient cords for duration electrodes	item number: 13585

**4.5  
Technical Data**

Name and model	: <b>MEDISANA TENS Pain Therapy TDD</b>
Dimensions	: app. 145 x 66 x 30 mm
Weight	: app. 150 g
Channels	: 2
Curve form	: balanced bi-phasic, mono-phas
Output	: constant current
Intensity	: in 20 steps from 0 to 80 mA
Frequency	: in steps, 2-5-10-20-40-60-80-120-200 Hz
Pulse width	: in steps, 50-100-150-200-250 µs
Operations forms	: constant, burst, programs
Power supply batteries	: 2 x 1,5 Volt, LR6-AA, or equivalent storage batteries
Serviceable life	: 40 h
Operating conditions	: 0 °C to +50 °C, 20 to 90 % relative humidity
Storage conditions	: -10 °C to +60 °C, 20 to 93 % relative humidity
Article number	: 86202
EAN number	: 40 15588 86202 1

**CE 0297**

**We reserve the right to make technical and design changes in the course of continuous product improvement.**

## 5.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to send in the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

### **MEDISANA AG**

Itterpark 7-9  
40724 Hilden

Tel.: +49 (0) 2103 / 2007-60  
Fax: +49 (0) 2103 / 2007-626  
eMail: [info@medisana.de](mailto:info@medisana.de)  
Internet: [www.medisana.de](http://www.medisana.de)

The service centre address is shown on the last page.

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D&L Marketing Oy
ALERE Electronics AB