MEDISANA®

Ultraschall-Inhalationsgerät USN Ultrasonic Nebulizer USN

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Nebulizador por ultra-sons USN

Ultrasoon inhalatieapparaat USN

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Ultraljuds-inhalationsapparat USN

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Thank you

Thank you for your confidence in us and congratulations on your purchase!

By purchasing the ultrasonic nebulizer **USN** you have acquired a quality product by **MEDISANA**.

To ensure that you are able to obtain the results you are looking for and can enjoy using your **MEDISANA** nebulizer for a long time to come, we recommend that you read with care the following information about using and caring for the device.

1.1 Important

 **Read the following safety information carefully before using the device and keep these instructions in a safe place for later use.**

- Use the device only according to its intended purpose as specified in the instruction manual.
- Do not use accessories that are not recommended by the manufacturer.
- Do not put any objects into any of the outlets. Avoid contact with pointed or sharp objects. The warranty will be invalidated if used for purposes other than those for which it is intended.
- Supervision should always be in place if the device is used by, on or near children or people who are handicapped and can only make limited use of the device. Medical devices are not toys!
- Do not attempt to repair the device yourself in the event of a malfunction. Not only will this invalidate the warranty but it may result in serious danger (fire, electric shock, injuries). Repairs should only be carried out by authorised service centres.
- Do not operate the device in the vicinity of anaesthetic mixtures that are inflammable in air or oxygen.
- Do not operate the device in the vicinity of high-frequency electromagnetic transmitters.
- Do not leave the device outside or in a humid environment. The device is not protected against the ingress of liquids.
- When cleaning, do not hold the whole device or the nebulizer chamber under running water. Take care to ensure that no water gets inside the device.

When operating the device with the power supply unit, please pay special attention to the following points:

- Ensure that the mains voltage shown on the nameplate matches that of your electricity supply before plugging the power supply unit into the mains.
- Only connect the power supply unit when the device is switched off.
- Do not reach for a device that has fallen into water. Disconnect the mains plug immediately.
- Do not use electrical devices whilst taking a bath or shower. Do not store electrical devices in places where they might fall into the bath tub or wash-basin.
- Do not use the device under blankets or pillows.
- Never use the device on soft surfaces such as a bed or sofa which could block the vents. Keep the vents free from dust, hair and the like.
- Keep the mains cable away from hot surfaces.
- Do not use the device if the supply cable is damaged, the power unit is faulty, the device is not working correctly, if it has been dropped or has fallen into water or has been damaged.
- Never carry, pull or twist the device by the mains cable and do not get the cable jammed.
- Only pull the plug out of the wall outlet with dry hands if the device has been turned off and is no longer in use and also when replacing parts of the device.

When using storage batteries, please pay special attention to the following points:

- Never use the device whilst taking a bath or shower.
- If liquid should get into the device, remove the rechargeable battery immediately and discontinue use. Contact your specialist dealer or the service centre.
- The **MEDISANA USN** works with particularly low-pollution Ni-MH rechargeable batteries that can be charged using the mains adapter.
- Contact your specialist shop or **MEDISANA** if the mains adapter is faulty with the result that the rechargeable batteries can no longer be charged. Do not use any other mains adapter since there is a danger of fire and the warranty will be invalidated.
- The fully charged storage batteries should be kept in a cool and safe place if the device is not going to be used for a longish period.

1.2

Important advice - for your health's sake

- Consult your family doctor before using the device if you have diabetes or other illnesses.
- Pregnant women should observe the necessary precautions and their personal circumstances; consult your doctor if necessary.
- Do not use the device on parts of the face where there are swellings, burns, inflammations, skin rashes, wounds or the skin is sensitive.
- The treatment should be pleasant. Stop using the device and consult your doctor if you feel pain or using the device is unpleasant.

2.1

Scope of delivery

Please check first of all to ensure the device is complete.
The following are included with the **MEDISANA USN**:

- 1 Inhaler with chamber cover
- 1 Mouthpiece
- 1 Joining piece with mask
- 1 Adults' mask with cord
- 1 Children's mask with cord
- 1 Bottle of inhalant
- 1 Power supply unit
- 1 Battery pack
- 1 Storage bag
- 1 Instruction manual

2.2

Important information about inhalation therapy

Living = breathing / Breathing = living

Often, diseases of the respiratory system or the lung are lengthy, painful and life restricting. Timely, effective therapies such as inhalation therapy with aerosols can be of great help and assistance.

A gas + particle mixture is defined as an aerosol; mist is an example of a natural aerosol. Aerosol therapy delivers medicines in an "air transported" form (e.g. nebulized) which are inhaled. The great advantage of this therapy (unlike taking tablets for example) is that the medicines are administered at the precise point where they will have greatest effect (via the oro-nasal-pharyngeal cavity into the lung) without passing through other organs:

1. The effect is noticeably rapid.
2. There is no (significant) effect or side effect on other organs.
3. Small quantities of medicines are used.

2.3

**Attention!
Special features
of the
MEDISANA USN*****Ultrasound technology***

An aerosol is generated in the ultrasonic nebulizer due to the vibrations of a piezoelectric crystal.

Thus, electric energy is transformed into mechanical oscillations which create droplets. The optimum diameter of the droplets is 1-5 µm (according to "Guidelines for the aerosol therapy of lung diseases", Lungenliga Schweiz). The droplets can only penetrate into the narrow branches of the upper and lower airways if they have a small diameter thus transporting and depositing the medicine where it is intended to be most effective.

**Important
information
when using
preparations/
medications*****Inhalant*****ATTENTION**

- **Consult your doctor before using any medicinal products or medicines to ensure that they are suitable for your needs or alternatively only use the medicinal products/medicine prescribed for you by your doctor.**
- **The MEDISANA USN ultrasonic inhaler is suitable for use with all common aerosol products (aerosols) with the exception of**
 - **Products with solid constituents (suspensions)**
 - **Viscous solutions (high viscosity)**
 - **Volatile oils**
- **Please do not use any volatile oils since this may lead to damage to the nebulizer diaphragm.**
- **Use only water-soluble products/medicines or those which have an alcohol base.**
- **Use only isotonic saline solution to dilute the inhalant; otherwise bronchial spasms may occur in certain circumstances.**
- **Do not use oil-based inhalants.**

Accessories

There is no provision for combining with other devices.
Use only the original accessories with this device since only then can safe operation be guaranteed.
Use a separate accessory set for each user to avoid cross-infection or the transmission of harmful substances.

Leakage safety

When pouring the inhalant into the nebulizer chamber, only fill it to the maximum mark. Otherwise there is danger that liquids may get into the electronics of the device.

During inhalation therapy, the device may be tilted to a maximum angle of 45 ° forwards, backwards and to the side without affecting nebulization and thus the success of the treatment.

The **MEDISANA USN** is leakage proof, i.e. no liquid leaks out even if you tip the device over. Do not, however, shake the device when the nebulizer chamber is full; there is a chance that liquids could get into the electronics and destroy the device.

3.1 Area of application

Possible areas of application for inhalation therapy are acute diseases of the airways such as.:

Colds

Inflammations of the paranasal sinuses

or chronic diseases such as.:

Bronchial asthma

Bronchitis

Before using the **MEDISANA USN**, discuss with your doctor whether inhalation therapy is suitable for you and which medicines or medicinal products you should use in addition to the length and frequency of use.

3.2 Operation with rechargeable battery and mains adapter

Please remove the protective foil before using the rechargeable battery for the first time. Press the narrow flaps on sides at the bottom the device inwards at the same time. The rechargeable will be loosened and can be taken out of the device. Remove the protective foil and push the rechargeable battery back into the device.

Charging Time

To charge the rechargeable battery, plug the mains adapter into the charging socket **2** first of all and then plug it into the mains socket. The charging display **3** lights up green. It goes out when the rechargeable battery is fully charged. Charge the battery pack overnight (at least 15 hours) using the power supply adapter to achieve full capacity for initial use, or if the device has been stored without being used for over 3 months. The normal time for charging the battery is 5 hours. The rechargeable battery can also be charged independently of the device. Take the rechargeable battery out of the device and plug in the mains adapter as described above.

3.3 Service life of the rechargeable battery

With a fully charged battery, the device can be used for approx. 18-20 minutes disconnected from the mains. This is sufficient for 1-3 uses (depending on the dosage and the length of use). As a rule, the inhalation phase using the **MEDISANA USN** is 8-10 minutes if the chamber is filled to its maximum of 6 ml = 6 cc due to the ultrasound nebulizer's high performance.

ATTENTION

Do not keep trying to switch the device on if the batteries are flat or this may lead to them being damaged. The battery display **4 switches from green to yellow when the batteries are flat and the device switches off automatically after a short time even if the chamber is still full of inhalant.**

3.4 Mains operation

If the batteries are flat or the equipment-on indicator lamp **4** changes from green to yellow during operation, the device can still be operated. Connect the mains adapter for this purpose. Power is supplied by the mains adapter during therapy. After use, leave the adapter connected to the mains so that the batteries are recharged.

3.5 Preparation for inhalation

Before operating for the first time or if the device has not been used for some time, clean and disinfect the nebulizer chamber as well as the mouthpiece and mask including the connecting piece (also refer to “Cleaning and maintenance”).

Remove the chamber cover **5** from the device and pour the inhalant into the chamber (maximum contents 6 ml = 6cc).

Please observe the dosage and instructions for use on the information leaflet enclosed with the medicine you are using.

If the inhalant is an essence, first fill the chamber with isotonic saline solution and then add the essence according to the dosage.

Close the chamber with the cover **5**. Make sure that the cover fits correctly and is tight. If necessary, turn the cover until you have found the right position. There is only one correct way of fitting the chamber cover to the device.

Then attach the mouthpiece **7** or the connecting piece and a mask onto the chamber cover.

3.6 Inhalation

IMPORTANT

Before starting therapy, discuss the length, dosage and frequency of use with your doctor. The method of use should be appropriate to the aim of the therapy (e.g. lung treatment = use of mouthpiece and treatment of the nasal-pharyngeal area = use of mask). Consult your doctor about this aspect as well.

3.7 Inhalation using the mouthpiece

1. Sit as upright as possible.
2. Enclose the mouthpiece completely with your lips.
3. Switch on the device by pressing the ON/OFF key **1**. The equipment-on indicator lamp (green, **4**) lights up. The aerosol warms up slightly during operation.
4. Now breathe in slowly and deeply through your mouth and out through your nose. You can improve the effectiveness of the therapy, if you hold your breath for a short time after breathing in. However, this is not necessary for general success of the therapy. It is only important that you keep quiet and relaxed while breathing in and out regularly. Do not inhale too fast. If you need a break, stop using the device for a short time and take the mouthpiece out of your mouth. Then put it into your mouth again completely (lips enclosing it tightly) and breathe slowly in and out again.
5. An inhalation application should not exceed 15 minutes.
6. After terminating the inhalation, switch off the device by pressing the ON/OFF key **1** even if you have not used up all the inhalant. The equipment-on indicator lamp (green, **4**) goes out.
7. The device switches off automatically after a short time (automatic limit stop) if the inhalant in the nebulizer is used up. For technical reasons a small amount remains in the nebulizer chamber.
8. Pour out the rest of the inhalant. Do not use it again.
9. Clean the device immediately after each use. (Also refer to "Cleaning and maintenance").

ATTENTION

Hold the device as upright as possible. A slight tilt does not affect its use (leakage proof). However, take care during inhalation not to tilt the device more than 45 ° in any direction.

3.8 Inhalation using the mask

Alternatively, two inhalation masks (for children and adults) and a connecting piece are supplied with the device. They can be used for inhaling aerosol through the nose. This method of use is especially recommended for treating children or (older) people with poor ability to co-operate.

1. Sit as upright as possible.
2. Place the connecting piece **8** between chamber cover **5** and mask **9**.
3. Tie the mask with the strings around your head or hold the mask pressed lightly over your nose. Make sure that the strings run above your ears and that the mask seals around the nose without being too tight.
4. Switch on the device by pressing the ON/OFF key **1**. The equipment-on indicator lamp (green, **4**) lights up. The aerosol warms up slightly during operation.
5. Now breathe in and out slowly and deeply through your nose. You can improve the effectiveness of the therapy if you hold your breath for a short time after breathing in. However, this is not necessary for general success of the therapy. It is only important that you keep quiet and relaxed while breathing in and out regularly. Do not inhale too fast. If you need a break, stop using the device for a short time and take off the mask. Then put it back on and breathe slowly in and out again.

ATTENTION

Hold the device as upright as possible. A slight tilt does not affect its use (leakage proof). However, take care during inhalation not to tilt the device more than 45 ° in any direction.

6. An inhalation application should not exceed 15 minutes.
7. After terminating the inhalation, switch off the device by pressing the ON/OFF key **1** even if you have not used up all the inhalant. The equipment-on indicator lamp (green, **4**) goes out.

8. The device switches off automatically after a short time (automatic limit stop) if the inhalant in the nebulizer is used up. For technical reasons a small amount remains in the nebulizer chamber.
9. Pour out the rest of the inhalant. Do not use it again.
10. Clean the device immediately after each use. (Also refer to "Cleaning and maintenance").

**NOTE**

A larger amount of aerosol remains on the mucosa of the nasal pharynx when using the mask than when using the mouthpiece.

3.9 Automatic limit stop

The device is equipped with an automatic limit stop: the device switches off automatically after a short time if the inhalant is used down to a very low residual quantity and there is hardly any or no liquid in the chamber.

Do not operate the device if the nebulizer chamber is empty.

4.1 Frequently asked question and answer

What can the device be filled with?

Use only water-soluble preparations/medicines or product that are alcohol based. All common inhalation solutions for ultrasound inhalation devices can be used. Do not use oil-based inhalants. Ask your doctor which preparations are suitable for you. More detailed information on the topics of "Aerosols", "Inhalation" or "Breathing Therapy" can be found in specialist books, or obtained from study groups and associations, or from your doctor.

4.2 Extra Tips from MEDISANA

The **MEDISANA USN** with its storage bag means you can take your ultrasound inhaler with you safely and cleanly when travelling. Due to its low noise level, you can easily use the inhaler at night or in poorly soundproofed hotel rooms without disturbing your neighbours. Especially when treating children, it has been noted that the low-noise function leads to a higher level of acceptance of use since it is considered less disturbing and frightening.

NOTE

If you have several inhaler users in one household, and thus several accessory sets, we recommend that they are clearly marked, e.g. with a permanent ink pen, to prevent any mix up.

4.3 Cleaning and maintenance

IMPORTANT

Carry out the cleaning instructions carefully after each use as described below.

- Remove the rechargeable battery before cleaning the device.
- Clean out the battery compartment using a cloth moistened with at least 75% alcohol and dry with a lint-free cloth so that no residues are left behind in the compartment.
- Clean the chamber cover **5** in the same manner.
- When cleaning, do not hold the whole device or the nebulizer chamber under running water. Take care to ensure that no water gets inside the device.
- In the case of infectious illnesses, the device must be disinfected every day or all attachments must be sterilised (it is recommended that this measure is performed regularly either in the morning or the evening at a fixed time). To do this, put the parts into boiling water for 10 minutes. Allow all parts to dry separately and uncovered on a cloth. Do not push the extension parts together whilst they are still wet.
- All extension parts may also be autoclaved with steam (at 1 bar maximum).
- This cleaning routine should also be carried out if the device has not been used for a longish period of time and prior to using for the first time.

- It is also possible to use alcohol or alcohol-based spray disinfectants. Make sure that no disinfectant residues are left behind to ensure safe inhalation when next used.
- Always make sure that the nebulizer chamber has dried completely.
- Clean the device's casing with a lightly moistened cloth that has been soaked in mild soapy water.

4.4 Disposal

Dispose of your old electrical unit in an environmentally acceptable manner when it has reached the end of its service life! Packaging is re-usable or may be re-cycled; metal parts must be disposed of as scrap metal; plastics, electrical and electronic components must be disposed of as electro-scrap.

Remove the batteries before disposing of the device. Do not dispose of batteries in domestic waste. Dispose of batteries as special waste or take them to a battery collection point in a specialist shop.

Consult your local council or your dealer for information about recycling.

4.5 Accessories for MEDISANA USN

Ask at your specialist dealer or call our hotline directly.

Accessory set = 1 children's mask
1 adult's mask
1 mouthpiece
1 connecting piece
1 inhalant bottle

Article number: 54050

Power supply unit

Article number: 54055

Battery pack

Article number: 54056

Chamber cover

Article number: 54057

Use only the original accessories with this device since only then can safe operation be guaranteed.

4.6**Technical
Data**

Name and model	: MEDISANA Ultrasonic Nebulizer USN
Dimensions (L x W x D)	: approx. 170 x 90 x 55 mm
Weight	: approx. 300 g
Electric power supply adapter	: AC 230 V / DC 13,5 V, 1 A
Rechargeable battery type	: Ni-MH
Rechargeable battery life	: approx. 500 x charging/discharging
Rechargeable battery charging time	: approx. 5 hours
Ultrasound frequency	: 1,7 MHz
Nebulizing output	: ~ 0,4 ml/min.
Aerosol portion of inhalation	: 60 % of particles < 5 µm
Average particle size	: 3,4 µm
Filling volume inhalant	: max. 6 ml = 6 cc
Working temperature	: 0 °C to +45 °C
Working conditions	: Humidity less than 85 % relative humidity, non-condensing
Storage temperature	: -10 °C ~ 35 °C 2 years -10 °C ~ 45 °C 6 months -10 °C ~ 55 °C 1 month -10 °C ~ 65 °C 1 week
Storage conditions	: Humidity less than 95 % relative humidity, non-condensing
Classification	: Device classified type BF  Classified according to MDD: IIa
Article number	: 54002
EAN number	: 4015588540028

CE 0297

We reserve the right to make technical and design changes during the course of continuous product improvement.

5.1 Warranty and Repair Conditions

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to send in the unit, please enclose a copy of your receipt and specify what the fault is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, evidence of the date of purchase must be provided by presentation of the sales receipt or invoice.
2. Defects in materials or workmanship will be remedied free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period for either the unit or the parts replaced.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper use, e.g. non-compliance with the instructions for use.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA AG

Itterpark 7-9

D-40724 Hilden

Tel.: +49 (0) 2103 / 2007-60

Fax: +49 (0) 2103 / 2007-626

eMail: info@medisana.de

Internet: www.medisana.de

The service centre address is shown on the last page.

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D&L Marketing Oy
ALERE Electronics AB