



**Omron Health Management
Software Ver 1.3
User's Help Manual**

Copyright © 2006-2010 Omron Healthcare Co., Ltd.
World Rights Reserved

TABLE OF CONTENTS

Before Using the Software

Important Safety Information	4
Intended Use	4

Operating Environment Requirement5

User Management Screen

Register new user	6
Edit a user	7
Delete a user	7
Search for a user on the user list	7
View user data	7
Change the title names on user list	7
Change sorting order or hide user list at startup	8
Create CSV file	8
Exit the screen	8

Blood Pressure Data Management Screen

Menu Area

Transfer the data from the monitor to the PC	9
Print the screen	9
Set the target blood pressure	10
Create CSV file	10
Create PDF file	11
Exit the screen	11

Data Display Area

Function Chart	11
Date range selection	12
Time range selection	12
Graph selection	12
View the data list	15
Enter data manually	17

Walking Data Management Screen

Menu Area

Transfer the data from the step counter to the PC	18
Print the screen	18
Set the daily goal	19
Set the startup screen	19
Set the starting date on Weekly graph	19
Create CSV file	20
Create PDF file	20
Exit the screen	20

Data Display Area

Date range selection	21
Graph selection	21
Event and step counter usage	22
View walking data	22

TABLE OF CONTENTS

Combination Data Management Screen

Menu Area

Print the screen	23
Create CSV file	23
Create PDF file	23
Exit the screen	24

Data Display Area

Date range selection	25
Walking graph selection	25
Time range selection	25

Transferring/Merging User Data26
---	------------

Uninstalling the Software29
--	------------

Before Using the Software

- Use this software and Help Manual under conditions described on the License Agreement applicable to this product.
- It is prohibited to use or copy all or part of this software and Help Manual without prior permission from Omron Healthcare Co., Ltd.
- The specifications of this software and the descriptions in this Help Manual are subject to change without prior notice.
- The screen images printed in this Help Manual may slightly differ from the images displayed by the software.
- Microsoft, Windows, Internet Explorer, Access, and Excel are registered trademarks or trademarks of Microsoft Corporation in the United States and other countries.
- Adobe, Adobe logo, Acrobat and Reader are registered trademarks or trademarks of Adobe Systems Incorporated in the United States and other countries.
- Other company names and product names used in this Help Manual are trademarks or registered trademarks of the respective companies. Trademark and registered trademark symbols such as ® and TM are omitted in this Help Manual.
- In this Help Manual, the Microsoft Windows 7 Operating System is abbreviated to “Windows 7”, the Microsoft Windows Vista Operating System is abbreviated to “Windows Vista”, the Microsoft Windows XP Operating System is abbreviated to “Windows XP”, and the Microsoft Windows 2000 Professional Operating System is abbreviated to “Windows 2000”. Also, Microsoft Internet Explorer is abbreviated to “Internet Explorer”, and Microsoft Excel is abbreviated to “Excel”.

Important Safety Information

Always consult your doctor. Self-diagnosis of measurement results and self-treatment are dangerous.

Intended Use

This product is designed to view the data transferred from the Omron blood pressure monitor or the Omron step counter on a PC. It is mainly designed for general household use.

Please read this Help manual, the blood pressure monitor's manual, the step counter's manual, and the PC's manual before using this product.

Operating Environment Requirement

The following operating environments are required to run this software.


Supported OS:	Microsoft Windows 2000 Professional (SP4) Microsoft Windows XP Home Edition (SP2) Microsoft Windows XP Professional (SP2) Microsoft Windows Vista Home Basic Microsoft Windows Vista Home Premium Microsoft Windows Vista Business Microsoft Windows Vista Enterprise Microsoft Windows Vista Ultimate Microsoft Windows 7 Starter Microsoft Windows 7 Home Basic Microsoft Windows 7 Home Premium Microsoft Windows 7 Professional Microsoft Windows 7 Enterprise Microsoft Windows 7 Ultimate
PC:	800 MHz or higher CPU, PC/AT compatible machine
Memory capability:	512 MB or more
Hard disc space:	32 MB or more*
Display:	Super VGA (800 × 600 pixel) or higher resolution video adapter and monitor
Others:	A CD-ROM USB connector (USB Revision 1.1 compliant) Adobe Acrobat Reader 5.0 or higher

NOTE: If the OS environmental requirements are above those listed, then additional resources may be required.

* Additional 200 MB is necessary in the case that Microsoft .NET or MDAC is not installed.

User Management Screen

This software can register multiple users and manage the data for their blood pressure measurements and walking. Data is controlled by user name.

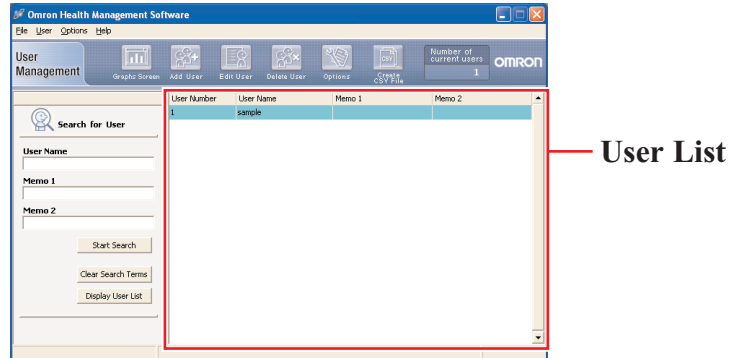

Register new user

Click the [Add User] icon then enter the user name and other information if necessary into the memo fields. Click [OK] to register.

User Name: sample
Memo 1: male
Memo 2: test user
OK Cancel Password Options

- NOTES:**
- A user name must be entered to register a new user.
 - The user name can be up to 28 characters, using either letters or numbers.
 - Up to 1000 unique users can be registered depending on the amount of free disc space in your PC.
 - The characters / \ : * ? " < > | , ; cannot be used in User Name.
 - The characters , ; cannot be used in Memo1 and Memo2.

Click [Password Options] on the dialog box to set a password if desired. Click [OK] to confirm.

User Name: sample
Current Password:
New Password: *****
Input again for confirmation: *****
OK Cancel

- NOTES:**
- Remember your password. If you forget your password you cannot log in, edit or delete the user. Write your password down, and keep it in a secure location.
 - To cancel the password leave the “New Password” and the “Input again for confirmation” fields blank.

User Management Screen

Edit a user

Select the user name on the user list then click the [Edit User] icon.

Delete a user

Select the user name on the user list then click the [Delete User] icon.


NOTE: To select multiple users click the desired users while pressing the Ctrl key or the Shift key.

Search for a user on the user list

Enter the keywords in “User Name”, “Memo 1” or “Memo 2” then click [Start Search].

Click [Clear Search Terms] to clear the keywords.

Click [Display User List] to display all users.

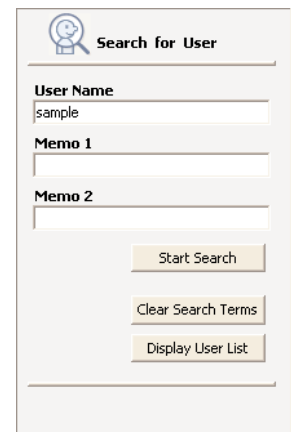

The 'Search for User' dialog box features a search icon and the title 'Search for User'. It contains three input fields: 'User Name' with the text 'sample', 'Memo 1', and 'Memo 2'. Below these fields are three buttons: 'Start Search', 'Clear Search Terms', and 'Display User List'.

View user data

Select the user name on the user list then click the [Graphs Screen] icon.

When you click on the [Graphs Screen] icon for the first time, the dialog box to select [Blood Pressure Monitor] or [Step Counter] will appear. Select the icon that represents the data you want to view.

The next time that you click on the [Graphs Screen] icon, it will automatically display the screen for the data type you viewed the previous time. If you wish to change which information you want to view, click on the [Change Screens] icon, and pick the appropriate selection.




Change the title names on user list

Click the [Option] icon then click the [Setting Heading Names] tab.

Enter title names if needed.

Click [OK] to confirm.

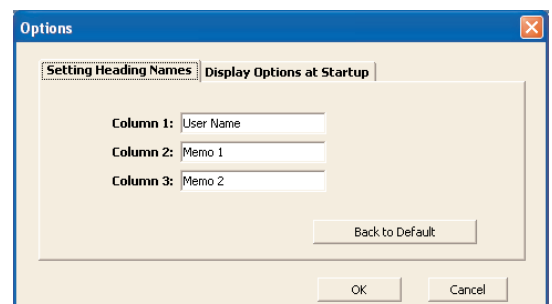

The 'Options' dialog box shows the 'Setting Heading Names' tab. It has two sub-tabs: 'Setting Heading Names' and 'Display Options at Startup'. Under 'Setting Heading Names', there are three input fields: 'Column 1: User Name', 'Column 2: Memo 1', and 'Column 3: Memo 2'. At the bottom, there are buttons for 'Back to Default', 'OK', and 'Cancel'.


User Management Screen

Change sorting order or hide user list at startup

Click the [Option] icon then click the [Display Options at Startup] tab.

Select sorting order on “Sorting order at startup” to change the order at startup.

Select “Do not Display” on “User List Display at Startup” to hide the user list at startup.

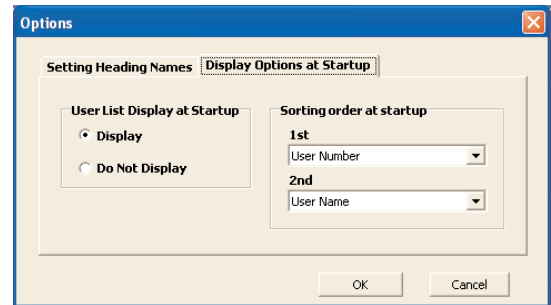


Create CSV file


This software can create a CSV file for export and database purposes for a single user or multiple users. CSV files can be used to bring data into other programs, such as EXCEL.

NOTE: To select multiple users click the desired users while pressing the Ctrl key or the Shift key.

1. Select single or multiple users on the user list then click the [Create CSV file] icon.
2. Click either [Blood Pressure Data] or [Step Counter Data].

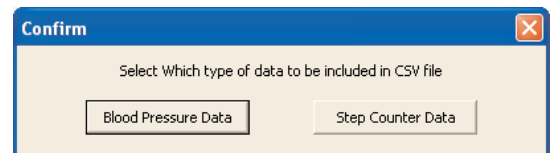

3. Select the destination folder and change the file name if necessary.

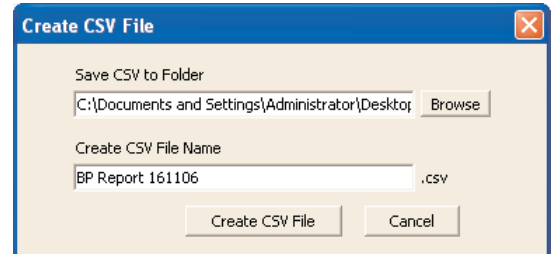

4. Click [Create CSV File] to save the file to the destination folder.

Exit the screen

Click [X] at the upper right of the screen. You will exit the Omron Health Management Software program.



Blood Pressure Data Management Screen



The Blood Pressure Data Management screen consists of the following major areas.



Menu Area

Transfer the data from the monitor to the PC

1. Connect the USB cable from the Omron blood pressure monitor to the PC. Click the [Download Data] icon.
2. The Confirm dialog appears. Click [Yes] to import the data.
When you transfer the data for the first time, the [Options] box as shown below will appear. Select the model of your monitor, and if your monitor is M10-IT, M9 Premium or i-Q142, select USER A or USER B of the monitor you use.

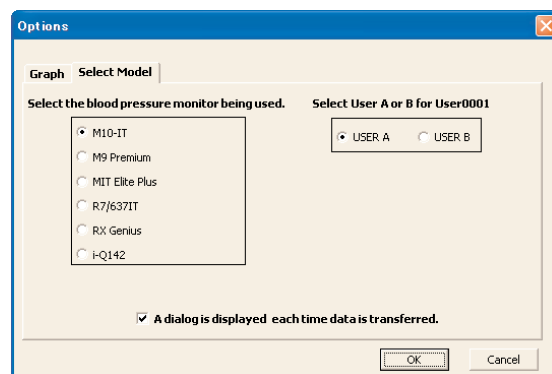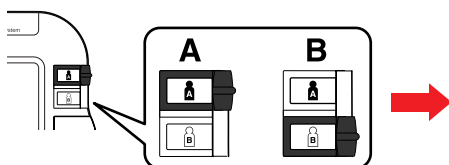

3. Start importing data. Click [Cancel] if you want to cancel.
4. The dialog box appears when complete. Click [OK].

NOTES:

- You cannot transfer the data while taking a blood pressure reading.
- Do not turn the PC off or unplug the USB cable while the data is being transferred.


Print the screen

Click the [Print] icon, then set up to print according to your needs.

Blood Pressure Data Management Screen

Set the target blood pressure

Click the [Options] icon.

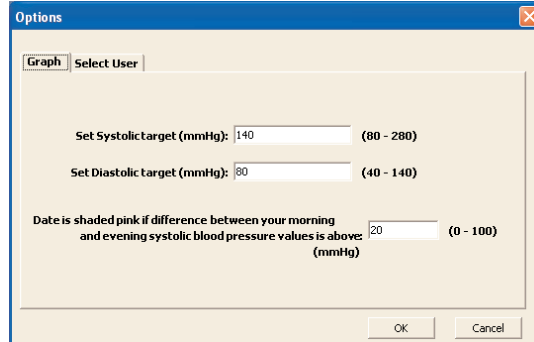

The 'Options' dialog box has a title bar with a close button. It contains two tabs: 'Graph' (selected) and 'Select User'. Under the 'Graph' tab, there are three input fields with their respective ranges in parentheses: 'Set Systolic target (mmHg):' with a value of 140 and a range of (80 - 280); 'Set Diastolic target (mmHg):' with a value of 80 and a range of (40 - 140); and 'Date is shaded pink if difference between your morning and evening systolic blood pressure values is above:' with a value of 20 and a range of (0 - 100). At the bottom are 'OK' and 'Cancel' buttons.

[Set Systolic target (mmHg)]

Enter your target systolic blood pressure value. An orange line indicating your systolic blood pressure target appears on the graph.

[Set Diastolic target (mmHg)]

Enter your target diastolic blood pressure value. An orange line indicating your systolic blood pressure target appears on the graph.

[Date is shaded pink if difference between your morning and evening systolic blood pressure values is above]

Enter the limit difference between morning and evening systolic blood pressure value. When the difference exceeds the limit, the corresponding dates are shaded pink on the graph.

NOTE: This function is available only in the case morning value is higher than evening value.


Click [OK] to set.

Create CSV file

CSV files can be used to bring data into other programs, such as EXCEL.

1. Click the [Create CSV File] icon.
2. Enter the date range and select the destination folder and change the file name if necessary.
3. Click [Create CSV File] to save the file to the destination folder.

NOTE: This CSV file is not available for importing CSV data (Refer to Page 26).

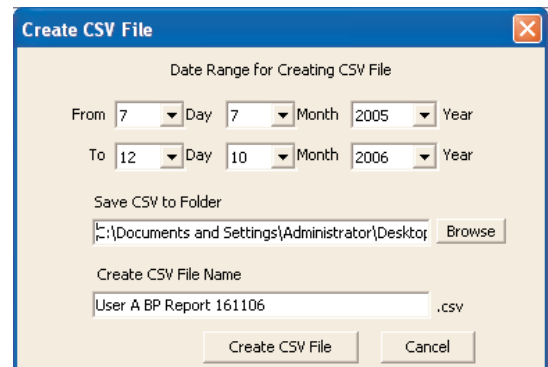

The 'Create CSV File' dialog box has a title bar with a close button. It contains a section 'Date Range for Creating CSV File' with 'From' and 'To' date pickers. The 'From' date is 7 Day 7 Month 2005 Year, and the 'To' date is 12 Day 10 Month 2006 Year. Below this is a 'Save CSV to Folder' section with a text box showing the path 'C:\Documents and Settings\Administrator\Desktop' and a 'Browse' button. The 'Create CSV File Name' section has a text box with the name 'User A BP Report 161106' and a '.csv' extension. At the bottom are 'Create CSV File' and 'Cancel' buttons.

Blood Pressure Data Management Screen


Create PDF file

PDF files can be used to email and/or print data.

1. Click the [Create PDF File] icon.
2. Enter the date range and select the destination folder and change the file name if necessary.
3. Click [Create PDF File] to create the PDF file.

Exit the screen

- A. Click the [Change Screens] icon and select the screen you wish to view.

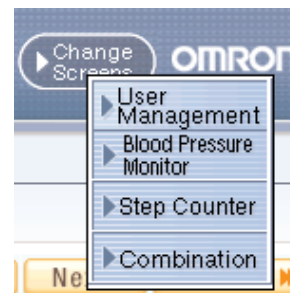

- B. Click [X] at the upper right of the screen. This returns you to the User Management screen.



Data Display Area

Function Chart

	[Weekly]	[Monthly]	[Yearly]	[All]	[Data List]
[Blood Pressure and Pulse]	✓	✓	✓	✓	
[Morning Evening Comparison]	✓	✓	✓		
[Frequency Distribution]	✓	✓	✓		
[Weekly Average]		✓	✓		
[Time Range Selection]	✓	✓	✓		
Data List	✓	✓	✓		✓
Comment Entry	✓	✓			
Manual Data Entry	✓				

Blood Pressure Data Management Screen

Date range selection




[Weekly]: Displays weekly data

[Monthly]: Displays monthly data

[Yearly]: Displays yearly data

[All]: Displays all data

[Data List]: Displays the data list selected. Input the data ranges you wish to view.

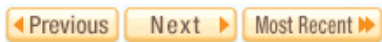

[Previous]: Displays the previous week/month/year data

[Next]: Displays the next week/month/year data

[Most Recent]: Displays the most recent week/month/year data

Time range selection



[Total]: Displays all of the data stored

[Morning]: Displays the blood pressure and pulse readings recorded between 4:00 AM-9:59 AM

[Daytime]: Displays the blood pressure and pulse readings recorded between 10:00 AM-6:59 PM


[Evening]: Displays the blood pressure and pulse readings recorded between 7:00 PM-3:59 AM

[Average]: Displays the average data for all readings

Graph selection




[Blood Pressure and Pulse]: Displays the trend of your blood pressure and pulse readings.

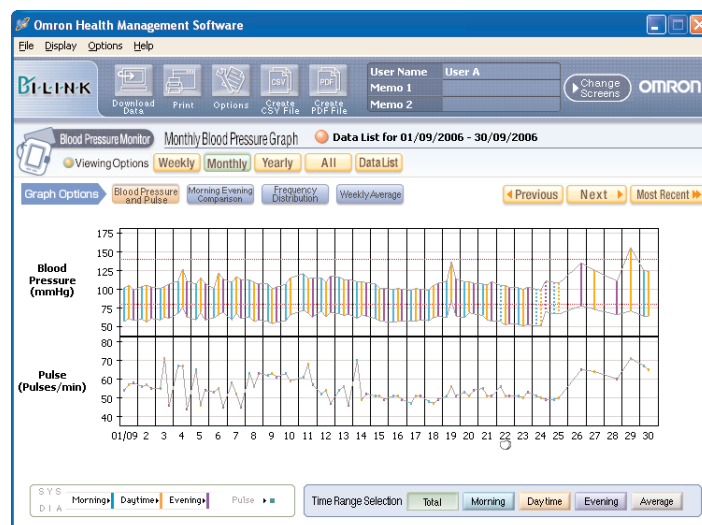


Blood Pressure Data Management Screen

How to interpret the graph.

- Upper Line: Systolic value
 Lower Line: Diastolic value
 Dotted vertical line: Includes manually entered data
 Solid vertical bar: Only includes data downloaded from the blood pressure monitor
 Bar and dots in Blue: The average value of all the blood pressure and pulse readings taken in the morning (4:00 AM-9:59 AM)
 Bar and dots in Orange: The average value of all the blood pressure and pulse readings taken in daytime (10:00 AM-6:59 PM)
 Bar and dots in Purple: The average value of all the blood pressure and pulse readings taken in the evening (7:00 PM-3:59 AM)
 Pink shading: The difference between morning and evening systolic blood pressure value exceeds the designated limit for that day (Refer to “Set the target blood pressure” on page 10).
 Heart symbol: At least one Irregular Heartbeat reading is included in the data. If you move your cursor over the heart symbol, a box below will appear showing the number of total blood pressure readings and irregular heartbeats recorded during that time period.

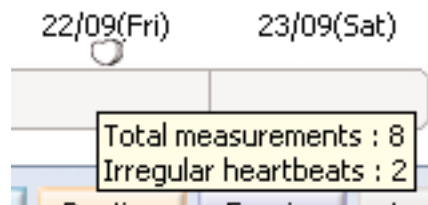


[Morning Evening Comparison]: Displays the average value of all the blood pressure and pulse readings taken in the morning and evening.

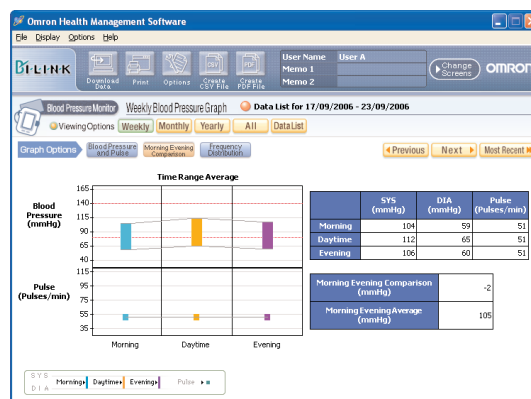


Blood Pressure Data Management Screen

[Frequency Distribution]: Displays the frequency distribution of the blood pressure and pulse readings.

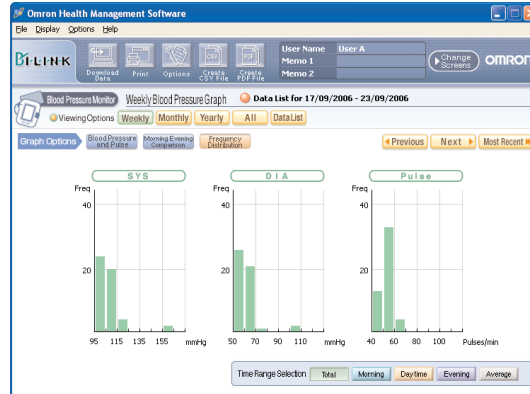

[Weekly Average]: Displays the weekly average value of the blood pressure and pulse readings. When you use the M10-IT, M9 Premium or i-Q142 monitor, the first 3 measurements in the morning and last 3 measurements in the evening are averaged. When you use the MIT Elite Plus, R7, 637IT or RX Genius monitor, all measurements in the time range are averaged.

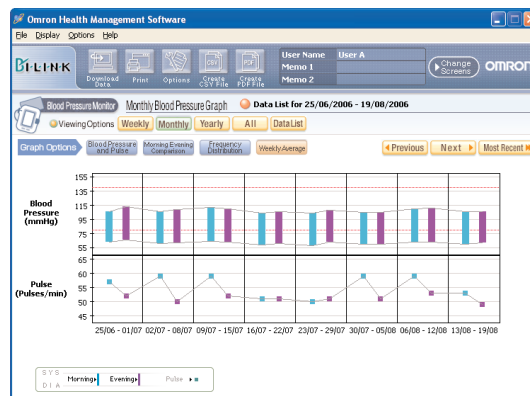

- NOTES:**
- If the data are imported as a CSV file, all measurements in the time range are averaged regardless of the monitor you use.
 - If one user uses 2 or more monitors, do not refer to the Weekly Average graph.

Blood Pressure Data Management Screen

View the data list

A. View the data list for a specific date range

Click the [Data List] icon then select the date range.

Search

Select the date range to be displayed.

Date Range : 07/07/2005 - 12/10/2006

From: 7 Day 7 Month 2005 Year

To: 12 Day 10 Month 2006 Year

OK Cancel

Click the [Search by Date] icon to change the date range.

Omron Health Management Software

User Name: User A

Memo 1

Memo 2

Search by Date

Hide	Date	Time	SYS (mmHg)	DIA (mmHg)	Pulse (Pulses/min)
<input type="checkbox"/>	12/10/2006	09:58	100	43	71
<input type="checkbox"/>	12/10/2006	09:58	90	44	69
<input type="checkbox"/>	12/10/2006	09:36	126	76	67
<input type="checkbox"/>	12/10/2006	09:34	106	62	66
<input type="checkbox"/>	11/10/2006	17:32	115	73	95
<input type="checkbox"/>	10/10/2006	10:10	100	70	70
<input type="checkbox"/>	09/10/2006	23:00	108	66	74
<input type="checkbox"/>	09/10/2006	23:00	108	65	74
<input type="checkbox"/>	09/10/2006	15:00	116	70	69
<input type="checkbox"/>	09/10/2006	15:00	116	70	69
<input type="checkbox"/>	09/10/2006	13:00	117	70	68
<input type="checkbox"/>	09/10/2006	13:00	117	70	68
<input type="checkbox"/>	09/10/2006	09:30	115	68	70
<input type="checkbox"/>	09/10/2006	09:30	115	68	70
<input type="checkbox"/>	09/10/2006	09:30	115	68	70
<input type="checkbox"/>	09/10/2006	09:00	115	80	70


Check the corresponding box on the “Hide” column if you do not want a certain reading to be displayed in the graphs.

Hide	Date	Time
<input checked="" type="checkbox"/>	12/10/2006	09:58

NOTE: “Hide” is available only for the [Blood Pressure and Pulse] trend graph.

B. View monthly data list

Click the number of the month (1 = January, 2 = February, etc) on the bottom of the Yearly graph to view the data list for that month.

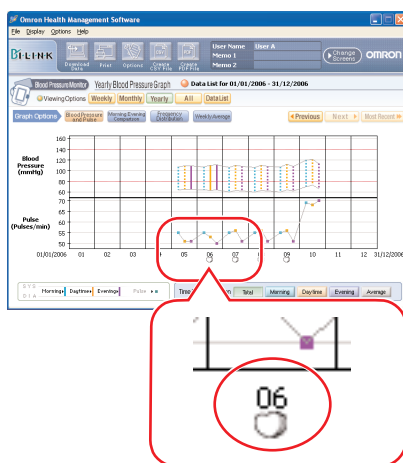

Blood Pressure Data

Blood Pressure Data for 06/2006

Date	Time	SYS (mmHg)	DIA (mmHg)	Pulse (Pulses/min)	Irregular Heartbeat
26/06/2006	20:28	100	63	45	
27/06/2006	09:08	105	59	53	
27/06/2006	09:09	103	58	54	
27/06/2006	16:06	112	71	52	
27/06/2006	16:06	107	60	51	
27/06/2006	20:52	114	67	46	
27/06/2006	20:53	109	66	45	
28/06/2006	09:33	108	67	67	
28/06/2006	09:34	103	62	67	
28/06/2006	17:03	113	65	44	
28/06/2006	17:03	110	62	48	
28/06/2006	19:55	112	60	44	
28/06/2006	19:56	113	63	43	
29/06/2006	09:41	107	60	55	
29/06/2006	09:42	108	61	56	
29/06/2006	14:17	148	90	81	
29/06/2006	14:18	147	90	81	

Close

Click the [Close] button at the bottom of the screen to close the data list.

Blood Pressure Data Management Screen

C. View daily data list

Click the date on the bottom of the Monthly or Weekly graph to view the data list for that day.


The screenshot shows the 'Omron Health Management Software' interface. On the left, a 'Weekly Blood Pressure Graph' is displayed with a date picker at the bottom. A red circle highlights the date '21/09(Thu)'. A red arrow points from this date to a 'List' window on the right. The 'List' window is titled 'Blood Pressure Data' and shows 'Blood Pressure Data for 21/09/2006'. It contains a table with the following data:

Date	Time	SYS (mmHg)	DIA (mmHg)	Pulse (Pulses/min)	Irregular Heartbeat
21/09/2006	08:59	106	66	56	
21/09/2006	09:00	108	63	54	
21/09/2006	15:12	108	60	52	
21/09/2006	15:13	104	58	50	
21/09/2006	20:14	111	57	49	
21/09/2006	20:15	108	57	53	

Below the table is a 'Comment' field and buttons for 'Close' and 'Save'.

In the Comment field, enter any information which you would like stored with the date shown, then click [Save] and [Close].


A circle appears in the Comment field under the date in which a comment was entered.

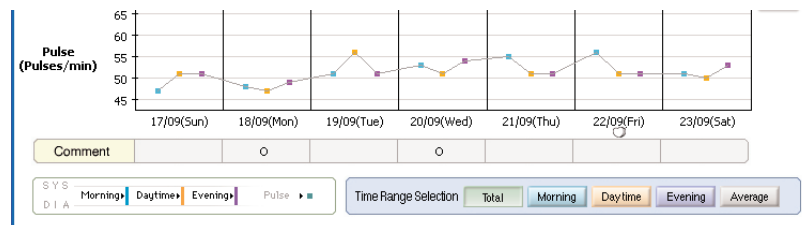

Blood Pressure Data Management Screen

Enter data manually

Click the [View/Enter Data] icon on the Weekly Blood Pressure and Pulse graph.

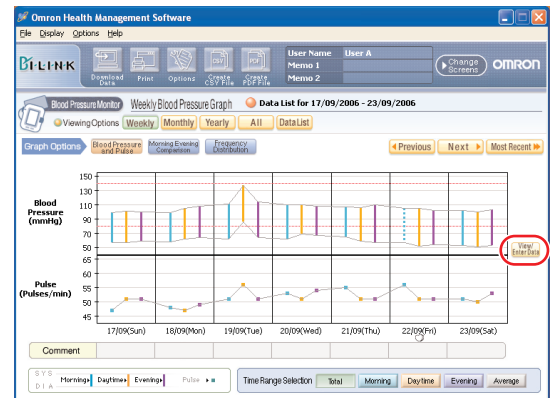

Enter your blood pressure and pulse readings at the bottom of the data list in the yellow area.

Check the box to enter an Irregular Heartbeat.

Click [Save] and [Close] to save the data.

Manually entered data is shaded pink in the data list.

Data downloaded from the blood pressure monitor is shaded blue or white in the data list. There is no distinction between blue and white shaded data; the two shade colors are used to simply make the data list easier to read.

To delete a row of manually entered data, leave all the columns blank, then click [Save] and [Close].

The screenshot shows the 'Blood Pressure Input' dialog box. It contains a table with columns: Date, Time, SYS (mmHg), DIA (mmHg), Pulse (Pulses/min), and Irregular Heartbeat. The last row is highlighted in yellow, indicating it is the row for manual data entry.


Date	Time	SYS (mmHg)	DIA (mmHg)	Pulse (Pulses/min)	Irregular Heartbeat
08/09/2006	09:54	112	62	60	
08/09/2006	09:55	111	63	65	
08/09/2006	14:55	107	57	55	
08/09/2006	14:56	111	57	56	
08/09/2006	20:57	103	57	62	
08/09/2006	20:58	110	61	63	
09/09/2006	08:58	107	55	62	
09/09/2006	08:59	106	59	61	
09/09/2006	14:59	100	54	60	
09/09/2006	15:00	101	54	65	
09/09/2006	21:00	104	56	62	
09/09/2006	21:01	101	55	60	
09/09/2006	22:00	120	80	60	

NOTES: • The data downloaded from the blood pressure monitor cannot be deleted, but it can be hidden.

- You will still be able to enter the data manually regardless of the period displayed in the graph.

Walking Data Management Screen

The Walking Data Management Screen consists of the following major areas.

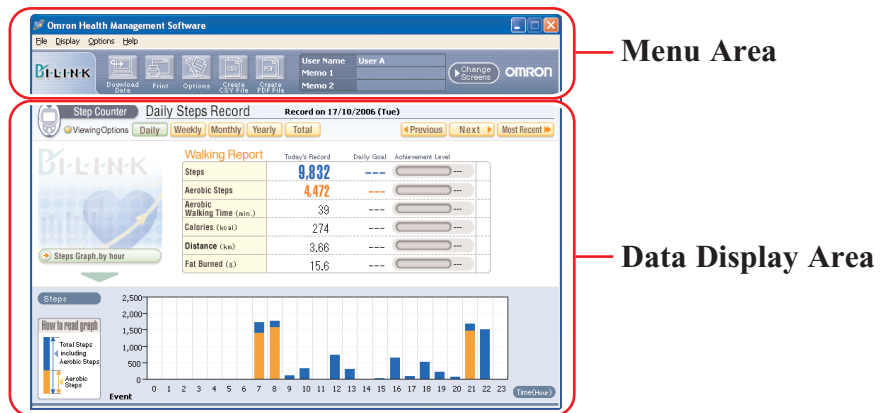

Menu Area

Transfer the data from the step counter to the PC


1. Connect the USB cable from the Omron step counter to the PC. Click the [Download Data] icon.

NOTE: The date shown by the software is determined by your PC. If you transfer the data at midnight, data may be stored as a different date. (e.g. 11:59 PM on the step counter and 12:00 AM the next day on the PC). To avoid this situation, it is recommended not to transfer the data at midnight, and to ensure the times on your PC and your Omron step counter are set correctly.

2. The Confirm dialog box appears. Click [Yes] to import the data.

NOTES:

- If you connect a different step counter from the last time you transferred the data, this screen shows up to confirm you still want to import and save the data.
- Data cannot be changed once it has been transferred.

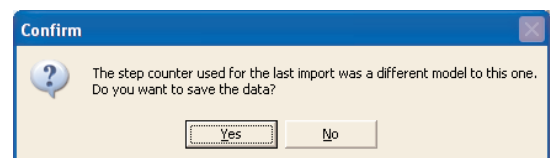

3. Start importing data. Click [Cancel] if you want to cancel.
4. The dialog box appears when complete. Click [OK].

NOTE: Do not turn the PC off or unplug the USB cable while the data is being transferred.


Print the screen

Click the [Print] icon, then set up to print according to your needs.

Walking Data Management Screen

Set the daily goal

Click the [Options] icon, then enter the numbers as instructed. The goal and achievement levels are displayed on the graph screen.

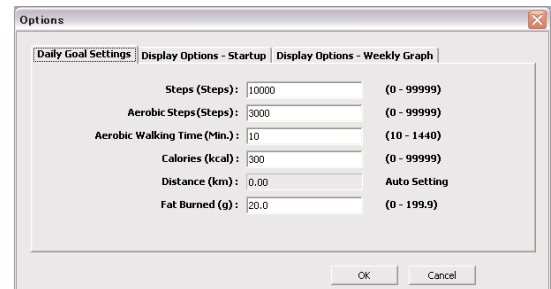

appears when the achievement level is between 0 and 75%.



appears when the achievement level is between 76 and 99%.



appears when the achievement level is 100% or higher.


∞ appears instead of percentage when the achievement level exceeds 1000%.

Click [OK] to set.

Set the startup screen

Click the [Options] icon, then select the [Display Options – Startup] tab.
Check the view and graph to be displayed at startup.


Click [OK] to set.

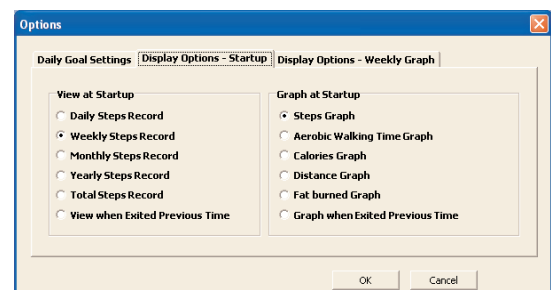

Set the starting date on Weekly graph

Click the [Options] icon then select the [Display Options – Weekly Graph] tab.
Check the starting date for the Weekly Graph.

Click [OK] to set.

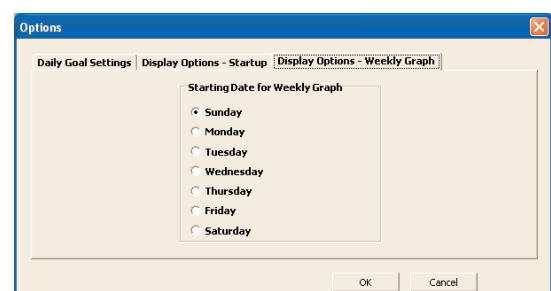


Walking Data Management Screen

Create CSV file

CSV files can be used to bring data into other programs, such as EXCEL.

1. Click the [Create CSV File] icon.
2. Enter the date range and select the destination folder and change the file name if necessary.
3. Click [Create CSV File] to save the file to the destination folder.

NOTE: This CSV file is not available for importing CSV data (Refer to Page 26).

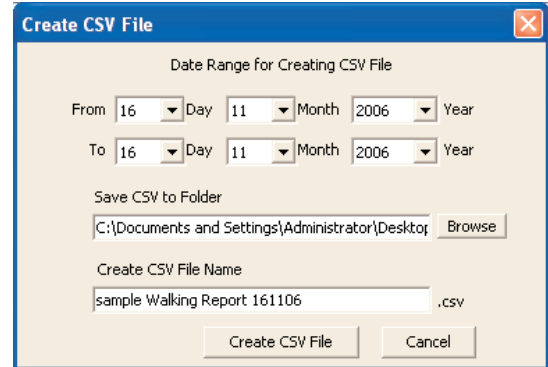

Create PDF file


PDF files can be used to email and/or print data.

1. Click the [Create PDF File] icon.
2. Enter the date range and select the destination folder and change the file name if necessary.
3. Click [Create PDF File] to create the PDF file.

NOTE: Hourly steps, hourly event and step counter usage (Refer to page 22) data is not exported to the PDF file.

Exit the screen

- A. Click [Change Screens] icon and select the screen you wish to view.

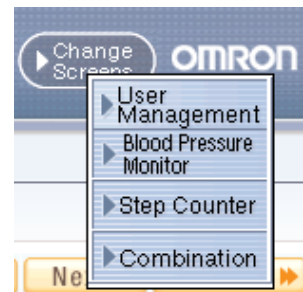

- B. Click [X] at the upper right of the screen. This returns you to the User Management screen.



Walking Data Management Screen

Data Display Area

Date range selection



- [Daily]: Displays daily graph with hourly steps
[Weekly]: Displays weekly graph with daily walking record
[Monthly]: Displays monthly graph with daily walking record
[Yearly]: Displays yearly data with monthly walking record
[Total]: Displays the total walking record and “Top 5” one-day total steps walked

NOTE: You can change the date range if you click the date or month on the bottom of the graph.
For example:

- Click the month on the bottom of the yearly graph → Monthly graph appears
Click the date on the bottom of the monthly graph → Weekly graph appears
Click the date on the bottom of the weekly graph → Daily graph appears



- [Previous]: Displays the previous week/month/year data
[Next]: Displays the next week/month/year data
[Most Recent]: Displays the most recent week/month/year data

Graph selection

Changing Graph




- [Steps Graph]: Displays total steps and aerobic steps
[Aerobic Walking Time Graph]: Displays aerobic walking time
[Calories Graph]: Displays calories
[Distance Graph]: Displays distance
[Fat Burned Graph]: Displays grams of fat burned

NOTE: Only the Steps Graph is available on the daily graph.

Walking Data Management Screen

Event and step counter usage

The daily graph displays 1) the hour when Event button on step counter was pressed, and 2) the hour when step counter was used.

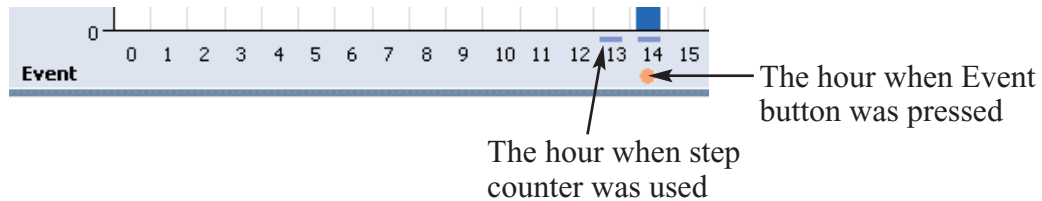


NOTE: If the step counter senses any movement during the hour, it records it as being in use even if no steps were taken.

View walking data

A. View monthly data

Click the bar graph of the month on the yearly graph.

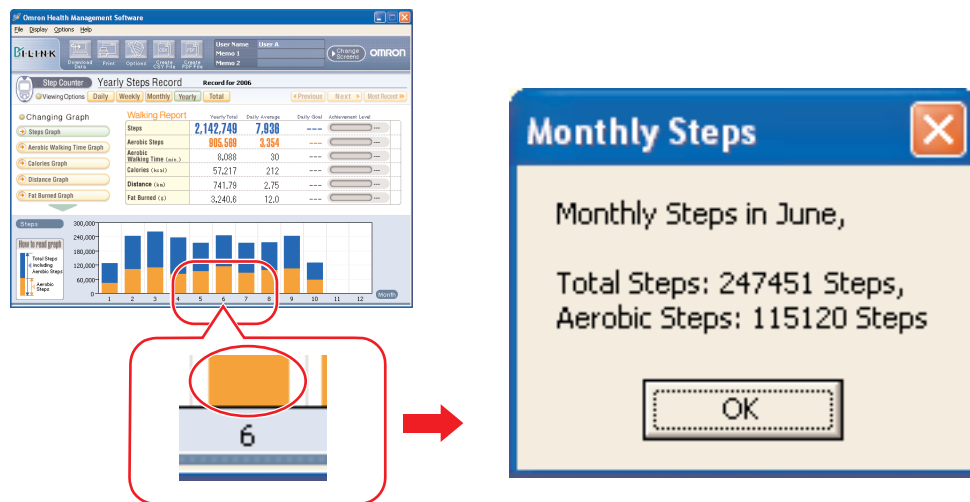

B. View daily data

Click the bar graph of the date on the monthly or weekly graph.

C. View hourly data

Click the bar graph of the hour on the daily graph.

NOTE: Only steps and aerobic steps are available for hourly data.

Combination Data Management Screen

The Walking Data Management Screen allows you to view blood pressure and walking data and graphs at the same time. This screen consists of the following major areas.



Menu Area

Print the screen

Click the [Print] icon, then set up to print according to your needs.

Create CSV file

You can export the date, systolic value, diastolic value, pulse, total steps and aerobic steps into a CSV file.

CSV files can be used to bring data into other programs, such as EXCEL.

1. Click the [Create CSV File] icon.
2. Enter the date range and select the destination folder and change the file name if necessary.
3. Click [Create CSV File] to save the file to the destination folder.

NOTES: • This CSV file is not available for importing CSV data (Refer to Page 26).
• Hourly steps, hourly event and step counter usage (Refer to page 22) data is not exported to the CSV file.

Create PDF file

You can export the date, systolic value, diastolic value, pulse, total steps and aerobic steps into a PDF file.

PDF files can be used to email and/or print data.


1. Click the [Create PDF File] icon.
2. Enter the date range and select the destination folder and change the file name if necessary.
3. Click [Create PDF File] to create the PDF file.

NOTE: Hourly steps, hourly event and step counter usage (Refer to page 22) data is not exported to the PDF file.

Combination Data Management Screen

Exit the screen

A. Click the [Change Screens] icon and select the screen you wish to view.

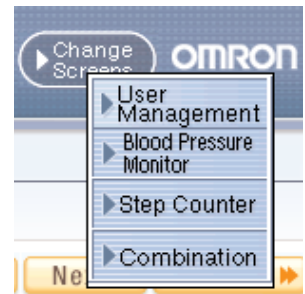

B. Click [X] at the upper right of the screen. This returns you to the User Management screen.



Combination Data Management Screen

Data Display Area

Date range selection


Viewing Options **Weekly** Monthly Yearly

- [Weekly]: Displays weekly graph with daily blood pressure/pulse readings and walking record
[Monthly]: Displays monthly graph with daily blood pressure/pulse readings and walking record
[Yearly]: Displays yearly graph with monthly blood pressure/pulse readings and walking record

◀ Previous Next ▶ Most Recent ▶▶

- [Previous]: Displays the previous week/month/year data
[Next]: Displays the next week/month/year data
[Most Recent]: Displays the most recent week/month/year data

Walking graph selection

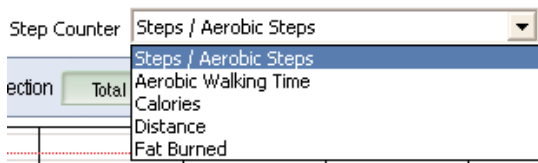

- [Steps/Aerobic Steps Graph]: Displays total steps and aerobic steps
[Aerobic Walking Time Graph]: Displays aerobic walking time
[Calories Graph]: Displays calories
[Distance Graph]: Displays distance
[Fat Burned Graph]: Displays grams of fat burned

Time range selection

Time Range Selection **Total** Morning Daytime Evening Average

- [Total]: Displays all of the data stored
[Morning]: Displays the blood pressure and pulse readings recorded between 4:00 AM-9:59 AM
[Daytime]: Displays the blood pressure and pulse readings recorded between 10:00 AM-6:59 PM
[Evening]: Displays the blood pressure and pulse readings recorded between 7:00 PM-3:59 AM
[Average]: Displays the average data for all readings

Transferring/Merging User Data


A. Import CSV data

Single or multiple user data from one PC can be transferred to another PC.

1. Go to the User Management screen.
2. Click on the user name(s) for whom you want to transfer data then click the [Create CSV File] icon.

NOTE: To select multiple users click the desired users while pressing the Ctrl key or the Shift key.

3. Click either [Blood Pressure Data] or [Step Counter Data]. Repeat steps 2. through 5. if you need to transfer both blood pressure and step counter data.
4. Select the destination folder and change the file name if necessary. If transferring both blood pressure and step counter data, use different file names.

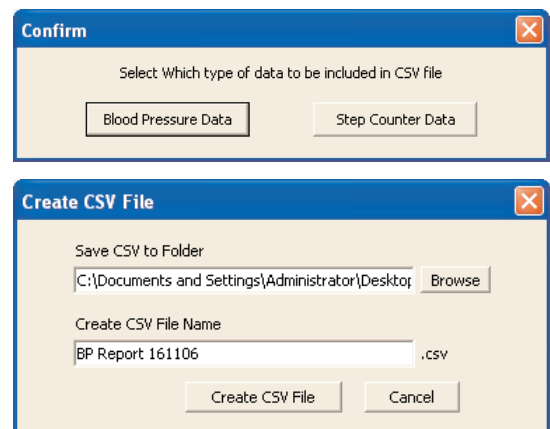

5. Click [Create CSV File] to save the file to the destination folder.

NOTE: CSV file is not created when there is no data.


6. Install the Omron Health Management Software in a second PC and move the CSV files to that PC.
7. Launch the Omron Health Management Software on the second PC.
8. From the User Management screen, click the [File] tab then select [CSV Import].



Transferring/Merging User Data


9. Select the CSV file for **blood pressure** data then click [Open].
If you do **not** need to import blood pressure data, click [Cancel].

NOTE: Select only the CSV file created by steps 1. through 5.

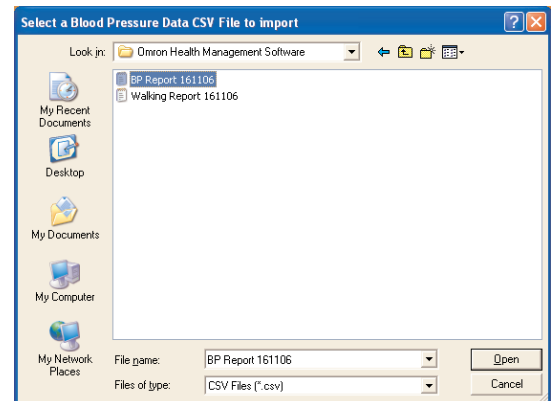


10. Select the CSV file for **step counter** data then click [Open].
If you do **not** need to import step counter data, click [Cancel].


NOTE: Select only the CSV file created by steps 1. through 5.

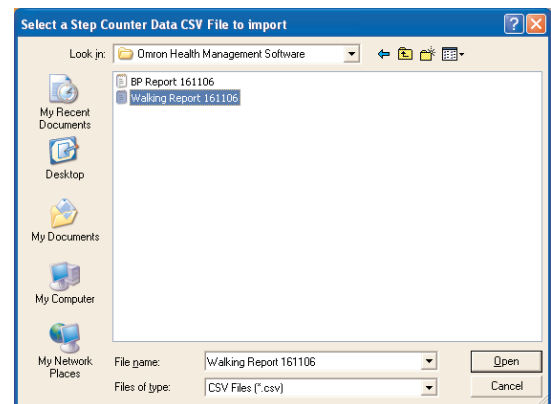

11. First the blood pressure data will be imported, followed by the step counter data.
Click [Cancel] if you want to cancel.

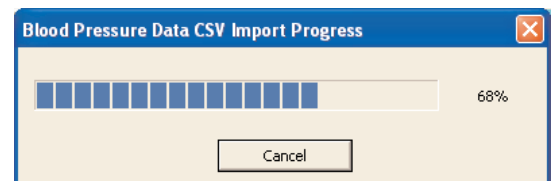

12. This dialog box appears when data transfer is complete. Click [OK].

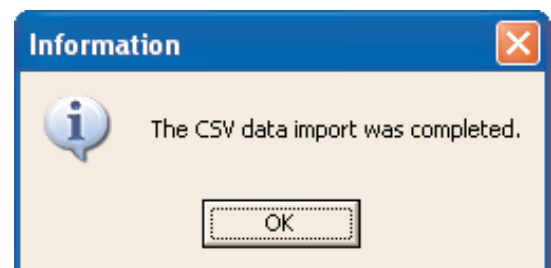

NOTE: Single or multiple user data from different PCs can be combined into one file for use on one PC by repeating steps 1. through 12. for each PC containing user data. Because all data is controlled by user name, duplicate user names can not be transferred into the same PC without losing the data from one or the other user name.


Transferring/Merging User Data

B. Transferring the Microsoft Access database file


Omron Health Management Software stores blood pressure and step counter data in one Microsoft Access database file named omron.mdb. This file is stored inside the [System] folder where the software was installed. This file can be transferred to another PC, however it will entirely replace the file on the receiving PC.

NOTE: If you want to merge user data from multiple PCs into one PC, follow procedure A. Import CSV data.

1. Open the [System] folder inside the folder in which you installed the Omron Health Management Software.

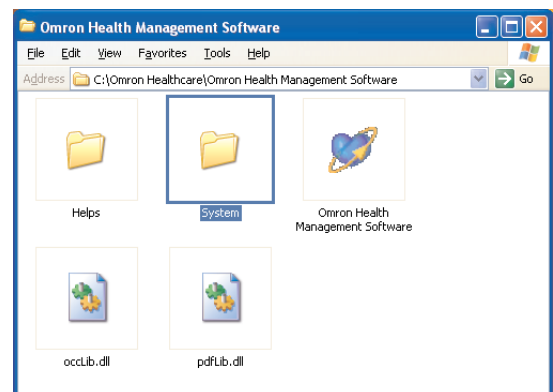

2. Copy and paste the [omron.mdb] Microsoft Access database file to a CD or other storage device.

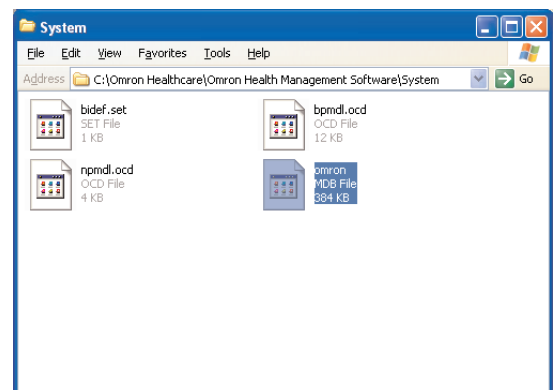



3. Install the Omron Health Management Software on the other PC.
4. Copy and paste the [omron.mdb] Microsoft Access database file from the storage device to the [System] folder located in the folder where you installed the Omron Health Management Software on the other PC.

Uninstalling the Software

1. Select [All programs] or [Programs] from [Start] on the PC.
2. Click [Omron Health Management Software].
3. Select [Uninstall Omron Health Management Software].
4. Follow the directions on the message boxes.

NOTES:

- When the software is uninstalled, all user data will be deleted.
- When you update the software, follow the direction for updated software installation.
- When the Operating System is Windows Vista or Windows 7 and the installation folder is [C:\Program Files\ OmronHealthcare\Omron Health Management Software], all data will not be deleted with the uninstallation. Please delete following folder to delete all.
C:\Users\User_name\AppData\Local\VirtualStore\Program Files\OmronHealthcare\
Omron Health Management Software (Underlined part is Windows login name.)

Manufacturer 	OMRON HEALTHCARE Co., Ltd. 24, Yamanouchi Yamanoshita-cho, Ukyo-ku, Kyoto, 615-0084 JAPAN
EU-representative 	OMRON HEALTHCARE EUROPE B.V. Kruisweg 577, 2132 NA Hoofddorp, THE NETHERLANDS www.omron-healthcare.com
Subsidiary	OMRON HEALTHCARE UK LIMITED Opal Drive, Fox Milne Milton Keynes, MK15 0DG U.K.
	OMRON MEDIZINTECHNIK HANDELSGESELLSCHAFT mbH John-Deere-Str. 81a 68163 Mannheim, GERMANY www.omron-medizintechnik.de
	OMRON Santé France SAS 14, rue de Lisbonne 93561 Rosny-sous-Bois Cedex, France